# SMOOV

## SMOOV one O10

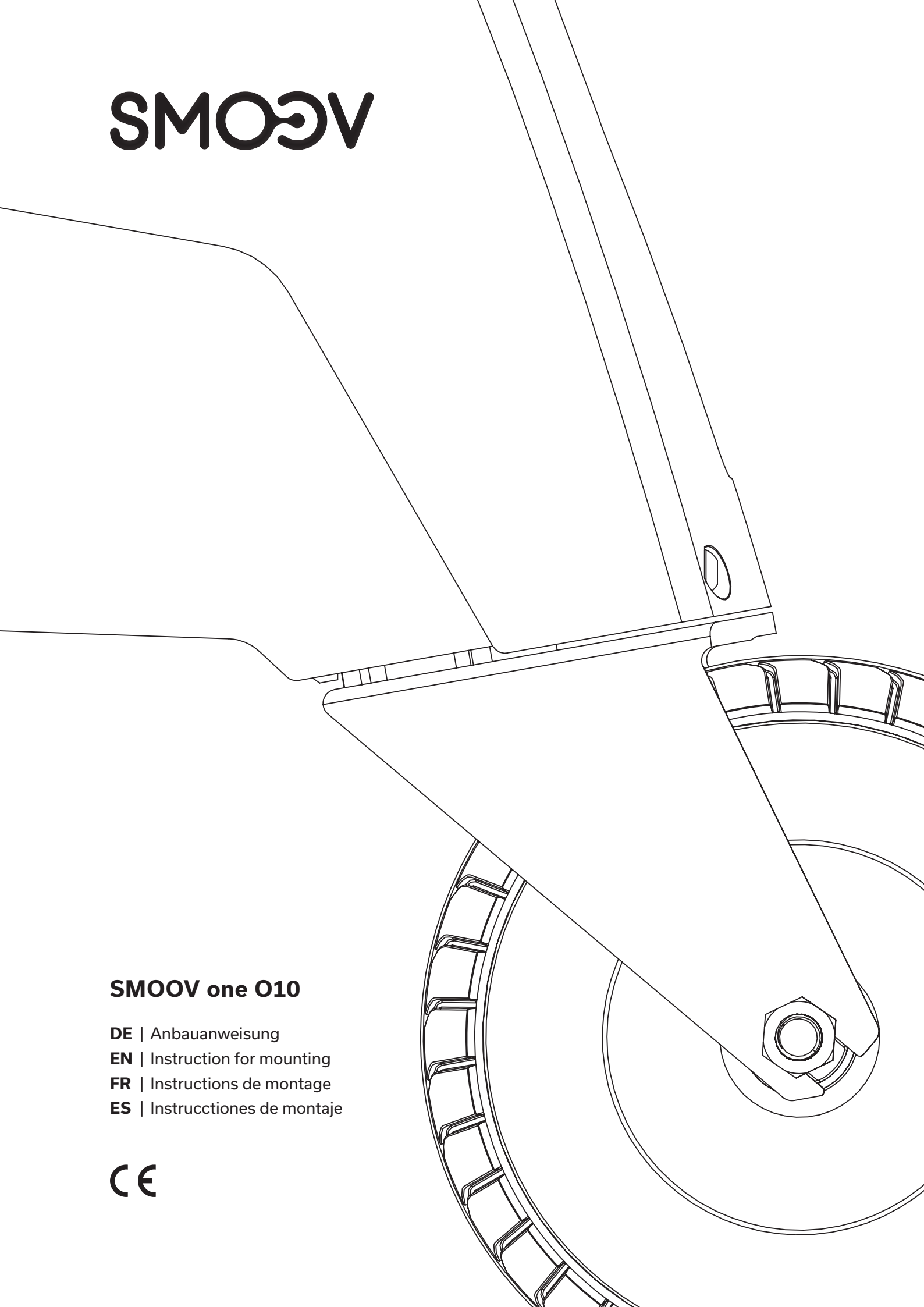
DE | Anbauanweisung

EN | Instruction for mounting

FR | Instructions de montage

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CE

45.0001.2.99.01  
Stand: 2019-02-12

## Wichtiger Sicherheitshinweis / *Important Safety Note* / Consigne de sécurité importante / *Indicaciones de seguridad importantes*

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*Work on the smooov, its individual components and the bracket on the wheelchair may only be performed by persons who have been authorised by Alber to perform this specific work.*

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---

### **Important information**

*These general installation instructions inform you about the basic installation steps to be observed when attaching the*

- *Bracket for the drive unit on a rigid frame wheelchair*
- *Adapter axle for the drive unit on a folding wheelchair*
- *Bracket for the control unit on all wheelchairs*

#### Please note the following in relation to rigid frame wheelchairs

*Specific attachment instructions for the customer's wheelchair are included in the delivery of the smooov. The information in these instructions regarding tightening torques and possible splices are to be observed and replace the information contained in these general installation instructions.*

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*The adapter axle [1] can only be installed on wheelchairs with protruding stub axles [A] and a wheel camber of not more than 3 degrees.*

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*In the diagrams of the installation work described below, the right-hand wheelchair wheel is not shown for reasons of presentation. Before starting the installation work, the wheelchair needs to be aligned, both wheels need to be fitted and the parking brakes need to be applied.*

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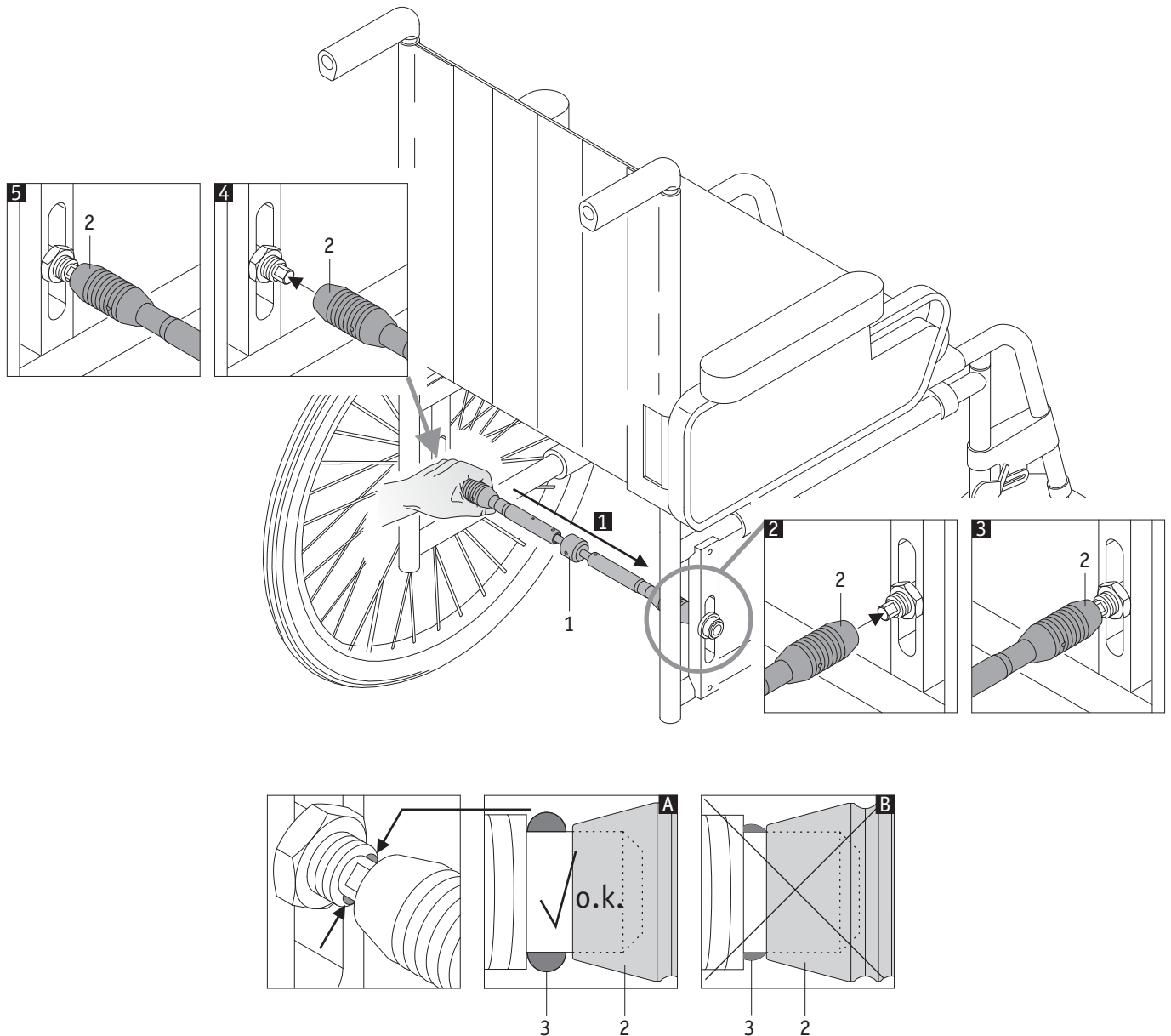
*Slide the two handles [2] of the adapter axle [1] over the stub axles of the wheels attached to the wheelchair.*

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
*The handles [2] must neither touch (A) nor press against (B) the retaining balls [3] of the wheelchair stub axle. If this is the case, the adapter axle [1] must be completely replaced.*

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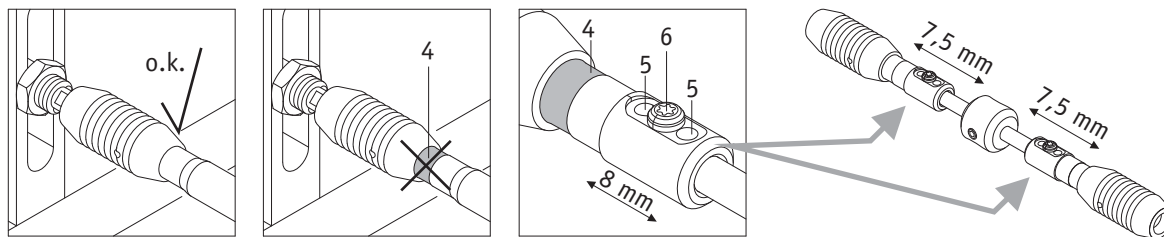*After attaching the adapter axle [1] to the wheelchair, it must no longer be possible to see the red marking [4] on either side. If this is the case, the length of the adapter axle needs to be changed. To this end, position the screw [6] in one of the holes [5] on each side.*

*If the adapter axle cannot be pushed over the stub axles of the wheelchair wheels when being attached to the wheelchair, the length of the adapter axle can also be adjusted in this way.*

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
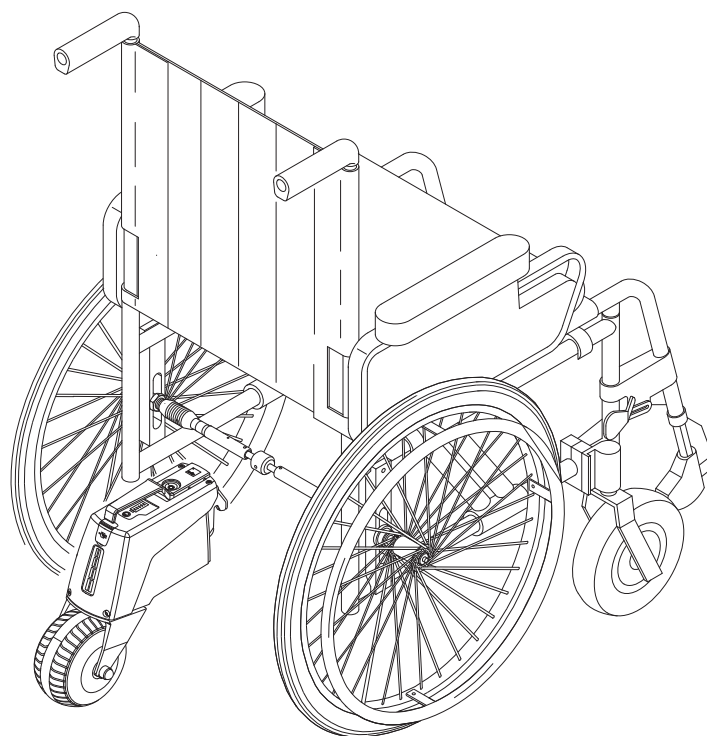
*Attach the drive unit of the smooV to the wheelchair in accordance with the instructions for use.*

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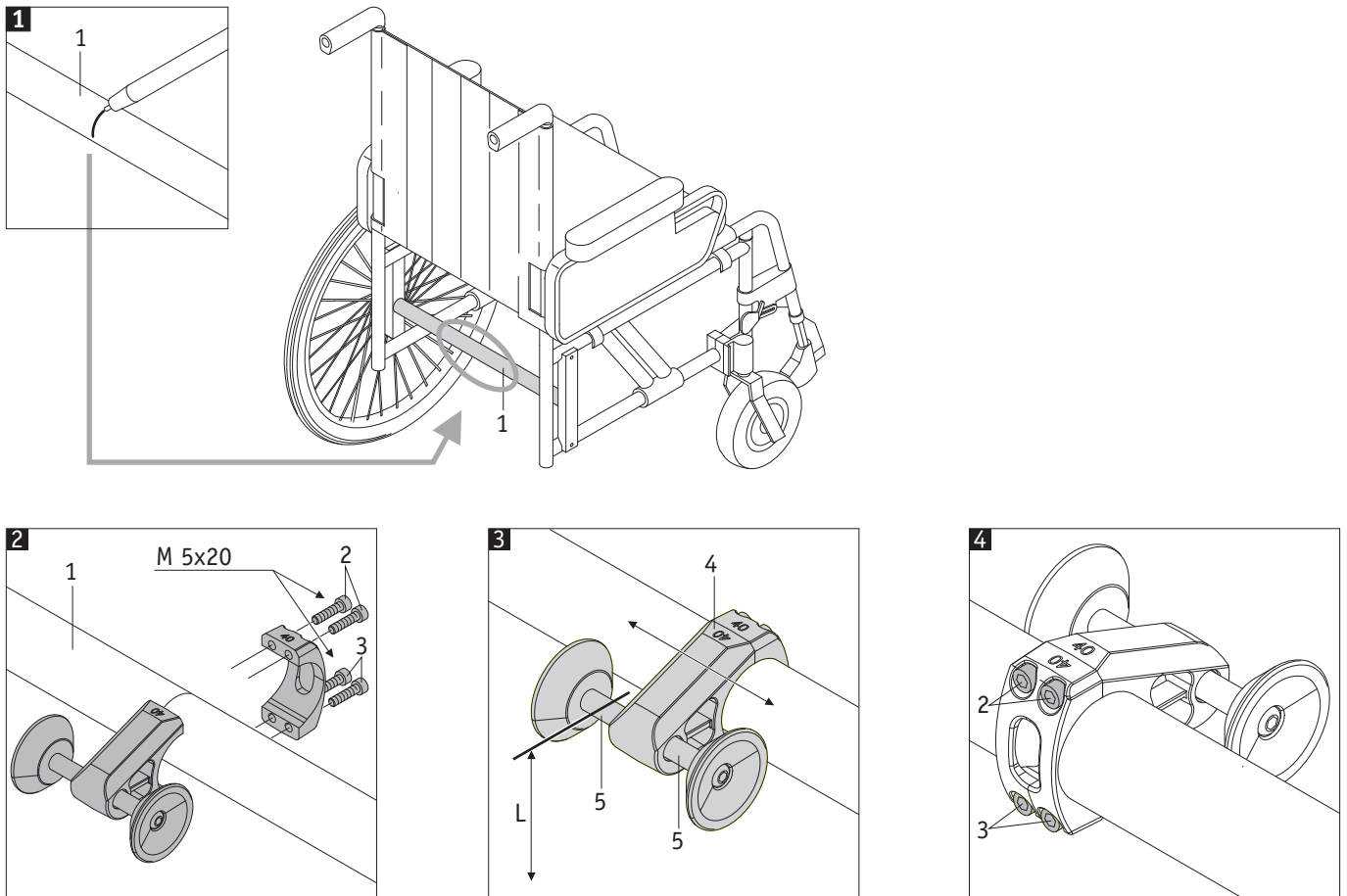
### EN

- 1** Mark the centre of the cross tube [1] on the wheelchair before starting installation work.
- 2** Attach the individual parts of the bracket in the area of the marking on the cross tube [1].  
Tighten the screws [2] with a torque of 5 Nm.  
Only tighten the screws [3] loosely so that it is still possible to move the bracket [4].
- 3** Check that the bracket [4] is correctly in position in the middle of the cross tube [1]. Correct the position if necessary.  
Then adjust the height alignment of the axle [5]. The distance L, measured from the top of the axle [5] to the ground, is 310 millimetres.
- 4** If the bracket [4] is centred on the wheelchair and if the height alignment is correct, the two screws [3] need to be tightened with a torque of 5 Nm. If the bracket is installed correctly, it must no longer be possible to turn it or move it!  
The drive unit of the smooov can now be attached to the wheelchair in accordance with the operating manual.
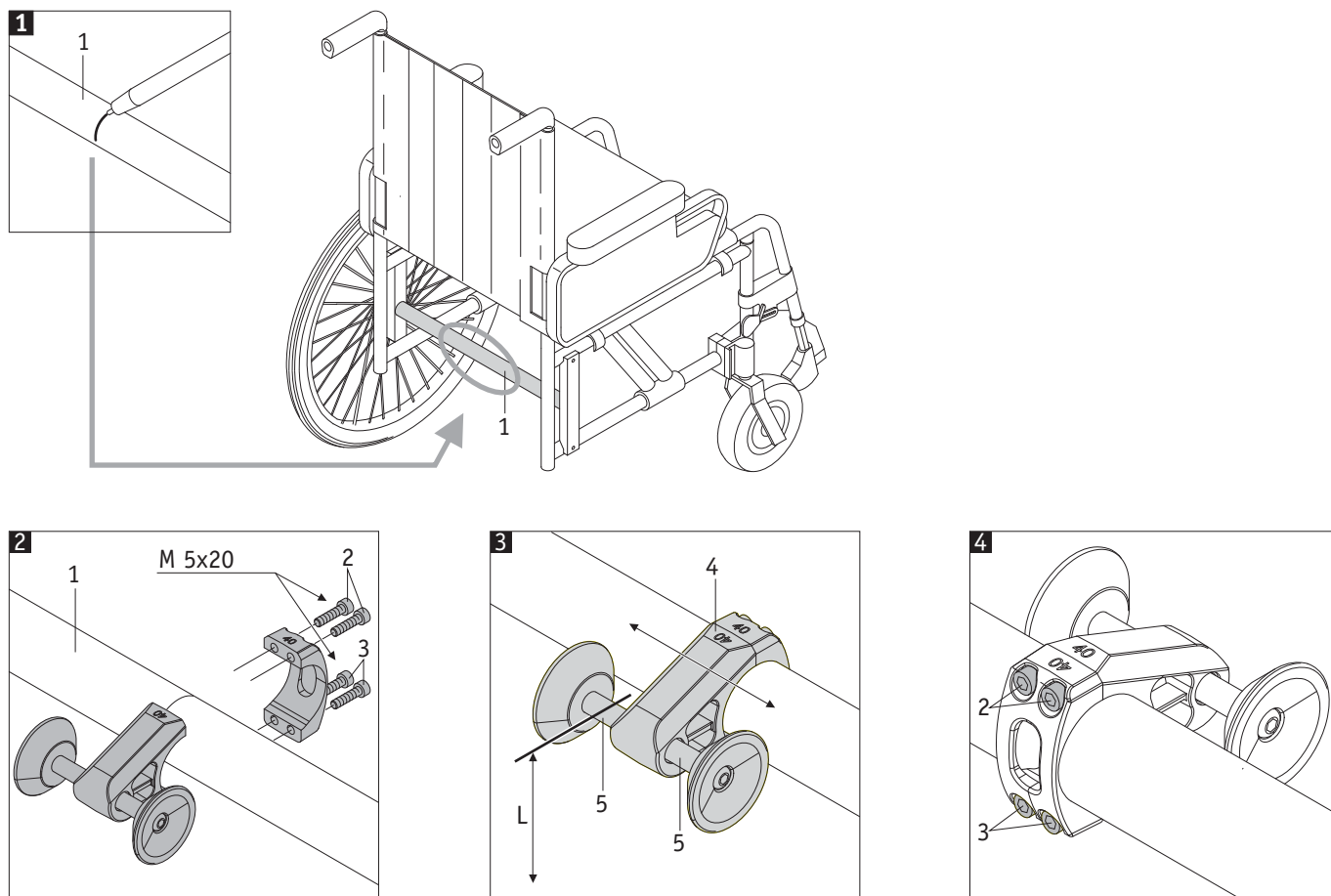
## Montage de la fixation pour les fauteuils roulants à cadre rigide / Montaje del soporte para sillas de ruedas con bastidor fijo

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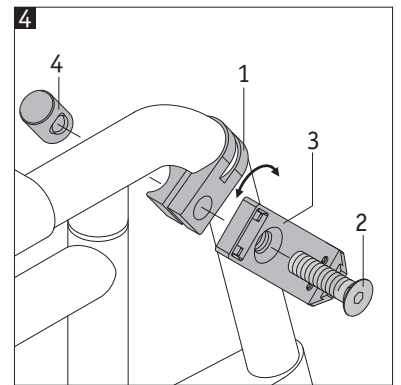
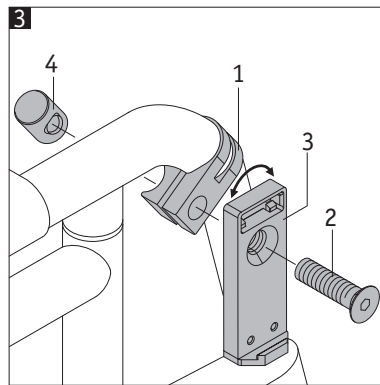
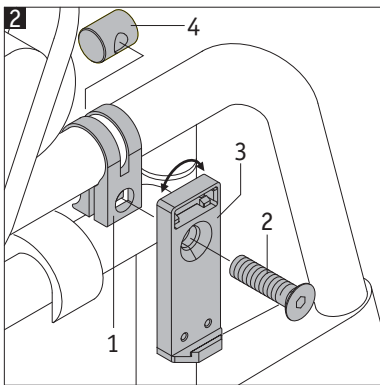
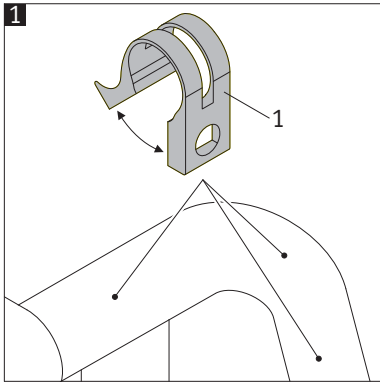
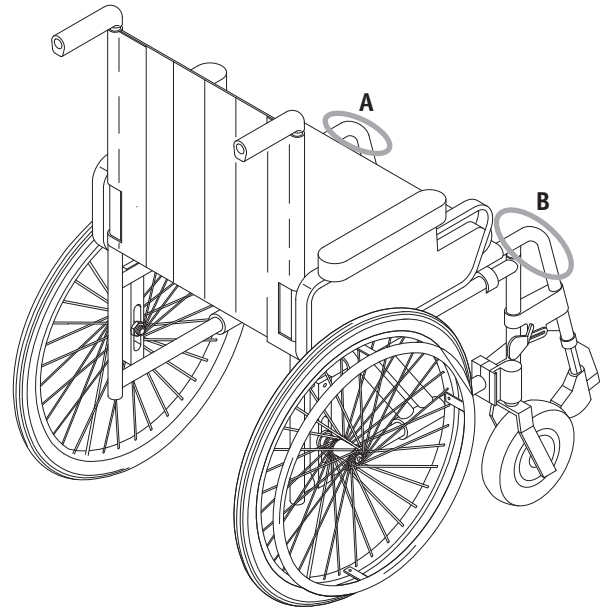
A = Possible installation position for a left-handed person  
 B = Possible installation position for a right-handed person

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### EN

**1** Spread the clamp [1] carefully and slide it over the frame tube of the wheelchair at the desired attachment position.

**2** Install the parts [1] to [4] as shown in the diagrams. In this regard, the retaining plate [3] can also be rotated axially to

**3** allow the control unit of the smooV to be positioned according to the wishes of the wheelchair user. If necessary, the LED

**4** displays of the control unit can be adapted accordingly. For details, see chapter 2.4 of the operating manual for the smooV.

**4** Screw tightening torque [2]: 12 Nm.

**If the bracket is attached to the left-hand side of the wheelchair (Pos. A), the control unit needs to be adapted to this position using the Mobility app. Please consult the operating manual for the Mobility app for further information.**

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**by Alber GmbH**