*Invacare® **Madison Plus***  
*Invacare® **Cleveland***  
*Invacare® **Miami***

<b>Guide d'utilisation</b>	<b>FR</b>
<b>User manual</b>	<b>EN</b>
<b>Gebruiksaanwijzing</b>	<b>NL</b>
<b>Manual del usuario</b>	<b>ES</b>
<b>Manuale d'uso</b>	<b>IT</b>
<b>Manual de Utilização</b>	<b>PT</b>
<b>Brugermanual</b>	<b>DA</b>

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Congratulations on your purchase of an Invacare® Lift-out chair.

We are sure it will provide you with optimum comfort every day.

This user manual is an integral part of the product and contains all necessary information for the correct use of the product to ensure it lasts a long time. This manual should be retained throughout the product's lifetime. We recommend that you contact us if you have any questions concerning spare parts. Recommendations for use of the product and all safety instructions are very important and must be followed without fail.

We recommend that you read this manual carefully before using the product for the first time.

This armchair is intended for home use. It helps the user to get up from a seated position to standing and vice versa and has a reclining function for enhanced comfort.

EN1

Stamp of Dealer

EN

# Summary

	Page
<b>1. General</b>	2
1.1 Symbols	2
1.2 Guarantee	3
<b>2. Security</b>	3
<b>3. Technical characteristics</b>	5
<b>4. Components and their functions</b>	5
<b>5. Assembly and installation</b>	6
5.1 Unpacking	6
5.2 Installation	6
5.3 Conditions for use	7
<b>6. Operation of the Lift-out chair</b>	7
<b>7. Maintenance and disinfection</b>	9
7.1 Maintenance instructions	9
7.2 Instructions for maintaining and cleaning the fabric	9
7.3 Daily maintenance	9
<b>8. Troubleshooting</b>	10
<b>9. Recycling</b>	10

EN2

## I. General

### I.1 Symbols

This symbol is used whenever a manoeuvre or situation is considered dangerous.

This symbol is used to indicate an important recommendation or point.

The “crossed-out dustbin” symbol is affixed to this product to encourage you to recycle it at selective collection points.

All advice and recommendations are important and must be complied with to ensure perfect operation of Invacare® Lift-out chairs.

## 1.2 Guarantee

Inspect the armchair to check for any damage, nicks, tears or damage to the frame. Inspect the remote control. If you notice any defects, do not attempt to use the armchair. Contact the distributor or carrier for instructions on how to proceed.

The guarantee provided by Invacare® runs for 24 months.

The manufacturer disclaims all responsibility if the instructions contained in this manual are not followed.

The following are excluded from the guarantee:

- damage resulting from accidents, falls or knocks;
- any irregular use of the armchair;
- damage related to liquids or other substances in contact with the armchair;
- normal wear resulting from use of the armchair.

The guarantee will have no effect if:

- the instructions in the user manual are not followed correctly;
- modifications or repairs to the armchair are carried out by a person not approved by Invacare®;
- indications point to incorrect maintenance of the armchair;
- indications point to the use of non-original parts.

## 2. Safety

The Invacare® Lift-out chair has been designed for use in the home.

All other forms of use shall be considered to contravene the recommendations we make below.

This armchair must only be used by an adult who has read the user manual beforehand.

The use of Invacare® Lift-out chairs is reserved for adults only.

To be used by one person at a time only.

Before installing or using the armchair, ensure that no person (more particularly a child) or animal is present within the area of operation of the armchair.

The leg rest folds in as it closes and could injure a child. Keep children's hands and feet well clear of the leg rest and the scissor or elevator mechanism.

 Ensure that arms and legs are not within the perimeter of the armchair's elevator mechanism or in contact with it. If it is necessary to remove objects that have fallen under the armchair, it is imperative to disconnect the power supply beforehand.

 Do not sit on the leg rest or armrests, they are not designed for this purpose and could be damaged; beware of the risk of falling.

 Do not try to sit in the armchair when it is in the raised position. Always leave the armchair in the seated position when not in use.

 Do not lean on the leg or backrest when sitting down, as this could cause the armchair to tip over.

 Do not carry out any cleaning or maintenance without disconnecting the power supply beforehand.

 Never direct a jet of water onto the armchair, this could cause a short-circuit.

 The Lift-out chair **MUST** be used with its seat cushion and backrest.

 Before carrying out maintenance or cleaning, please take all necessary precautions and disconnect the power supply.

 If any part of the armchair should catch fire, only use an approved powder type fire extinguisher.

 The covering of this product is not designed to be fire-resistant.

- Do not smoke in this armchair if you have difficulty getting up from the chair or if your mobility is impaired.  
Never place the armchair close to a major source of heat or ignition: (fireplace, electric heater, gas heater, oil heater, etc).  
- Do not expose the armchair to direct sunlight (UV) for a prolonged period of time.

 Always position the lift-out chair on a completely flat surface to ensure optimal stability during use.

 Protections of headrest and armrests (available as an option) have to be used to protect Lift-out chairs with vinyl cover in case of strong acid sweating.

### 3. Technical characteristics

Madison Plus & Cleveland = 1 motor

Miami = 2 motors

Voltage : 230 /240 Volts

Frequency : 50/60 Hz

Power : 50 w

Classe II


IP20

Fabric material : polyester

Foam density : 30 kg/cm<sup>3</sup>

	Madison Plus Miami Cleveland	110 kg 120 kg 120 kg		Madison Plus Miami Cleveland	105 cm 107 cm 102 cm
	Madison Plus Miami Cleveland	51 kg 53 kg 56 kg		Madison Plus Miami Cleveland	82 cm 82 cm 86 cm
	Madison Plus Miami Cleveland	50 cm 50 cm 52 cm		Madison Plus Miami Cleveland	83 cm 94 cm 93 cm
	Madison Plus Miami Cleveland	48 cm 49 cm 52 cm		Madison Plus Miami Cleveland	170 cm 183 cm 160 cm
	Madison Plus Miami Cleveland	53 cm 46 cm 56 cm		Madison Plus Miami Cleveland	64 cm 67 cm 64 cm
	Madison Plus Miami Cleveland	108/150° 104/175° 104/130°			

### 4. Components and their functions

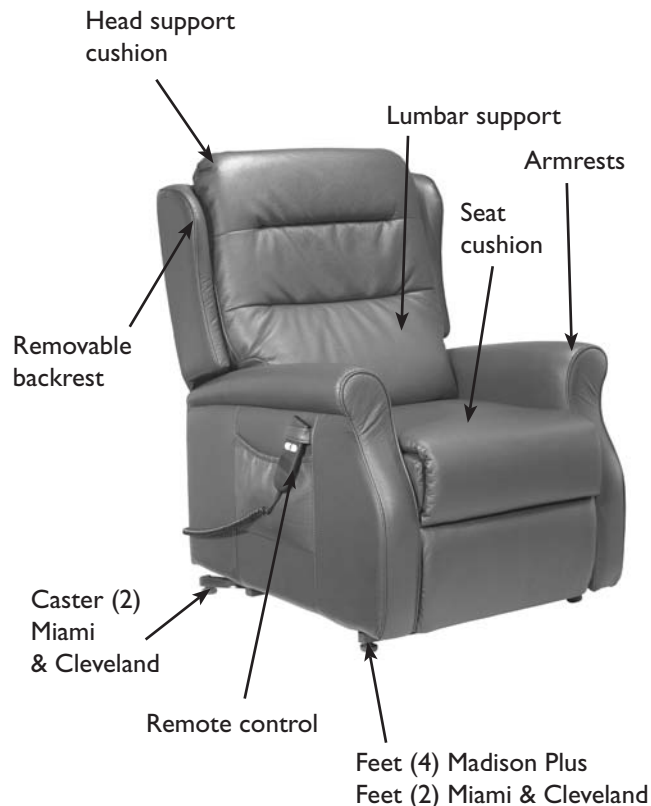## 5. Assembly and installation

### 5.1 Unpacking

 Open the cardboard box carefully when installing (do not use sharp objects when opening).

The cardboard packaging protects the Lift-out chair during transport:

- open the flaps of the cardboard box;
- remove the armchair's unassembled backrest;
- tear open the cardboard box at the 4 corners to remove the armchair;
- remove the protective plastic film and the anti-humidity sachets.

 Inspect the armchair to check for any damage, nicks, tears or damage to the frame. If you notice any defects, do not attempt to use the armchair. Contact the distributor or carrier for instructions on how to proceed.

 Remember to recycle all packaging in appropriate containers. Keep packaging out of reach of children at all times.

### 5.2 Installation

 Leave sufficient room (approximately 50 cm) behind the armchair to prevent it from coming into contact with the wall when in the reclined position.



 Make sure the armchair is positioned on a flat level surface.

 Do not place your armchair near a staircase.

 Never position the armchair near a major source of heat: (fireplace, electric heater, gas heater, oil heater, etc). Do not expose the armchair to direct sunlight (UV) for a prolonged period of time.

#### 1) Assembly

- Remove the transformer from its cardboard box held in place by a plastic ring under the frame of the chair.
- Connect the power supply cable to the transformer plug (A).
- Connect the mains cable (B) to the transformer and connector, and then connect the mains cable plug to the electrical power supply in compliance with the applicable standards and in accordance with the power supply unit voltage.
- Remove the plastic tie that attaches the chair power cable to the remote control.
- Position both backrest runners simultaneously in the mounting brackets until the backrest locks firmly in place (an audible click will be heard).

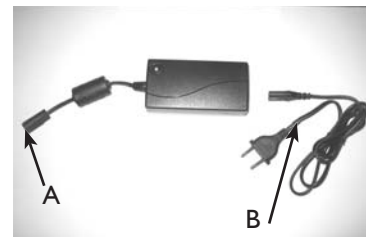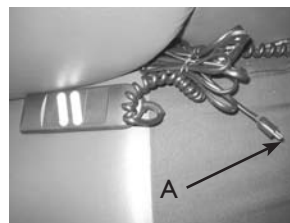- Place the Velcro strips of the rear skirt on the base of the frame.


 For your safety, assure you of the good locking of the backrest on the frame and verify the assembling by firing firmly the backrest. If this last is staying in position, you can use your lift-out chair.

Otherwise, restart the procedure of assembling of the backrest. If you doubt the good locking of the backrest, do not use the lift-out chair and contact your distributor, there is a risk of fall in case of bad assembling of the backrest.

**Nota:** To disassemble, position the backrest horizontally, pull outwards on the 2 tabs (A) and take out the backrest.

## 2) Readyng the armchair for use

 Always position the lift-out chair on a completely flat surface to ensure optimal stability during use.

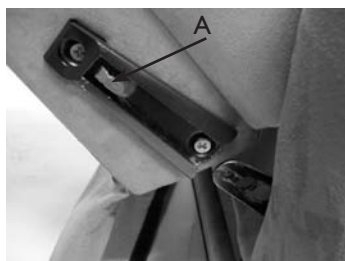## 5.3 Conditions for use

The Lift-out chair has been designed for use in the following conditions:

	Min	Max	Optimal
Temperature	5°C	40°C	20°C
Humidity		90%	60%

 Use of the armchair is not recommended if there is a risk of storms.

## 6. Operation of the Lift-out chair

EN7

 The covering of this product is not designed to be fire-resistant.  
Do not smoke in this armchair if you have difficulty getting up or if your mobility is impaired.

1) Do not operate this device without first reading and understanding this manual. If you have difficulty understanding the warnings and instructions, contact your reseller before using this equipment. Failure to do so may result in injury or material damage.

2) **WEIGHT LIMIT:** this Lift-out chair cannot be used by persons weighing more than **110 Kg (17.32 stone)** (Madison Plus); **120 kg (18.9 stone)** (Miami & Cleveland).

If the motor no longer works and the armchair is in the reclined position, it is imperative that the user should be assisted when getting out of the armchair.

 Always be sure to centre your weight on the seat cushion regardless of whether or not the armchair is reclined.

3) If the motor is used continuously for 2 minutes or more, wait 18 minutes before using it again.

**EN8** 4) Do not move the armchair when it is in the reclined position.

5) Do not move this armchair when it is in use or connected to the power supply.

If any part of the armchair should catch fire, use only an approved powder type fire extinguisher.

6) The remote control is intended for use only by the occupant of the armchair.

6.1) Chair with one motor (Madison Plus & Cleveland version)

Chairs with a single motor have a two-button remote control

A. Switch from a seated position to the lift-out function.

Return from the recline position to the seated position.

B. Switch from the seated position to the recline position.

Return from the lift-out function to the seated position.

6.2) Two-motor chair (Miami version)

Chairs with two motors have a four-button remote control for activating the backrest and leg rest separately, offering a wider variety of positions.

A. Switch from a seated position to the lift-out function.



Return from the recline position to the seated position.

B. Raise the backrest

C. Switch from the seated position to the recline position.

Return from the lift-out function to the seated position.

D. Lower the backrest.

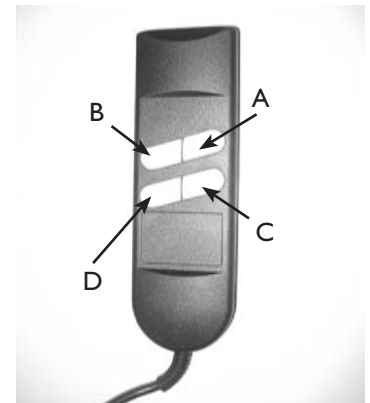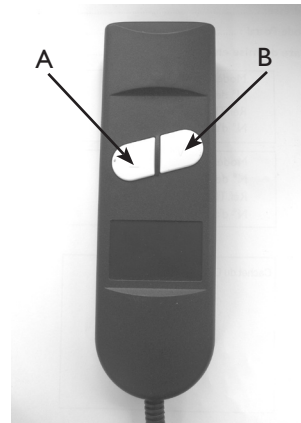## 7. Maintenance and disinfection

 Only authorised persons may repair or dismantle the product.

 Check that a person or pet is not located under or next to the frame before dismantling.

 To access the frame, tip the armchair onto its side and work from below.

Follow the instructions outlined below.

 Before carrying out any maintenance or servicing work, be sure to clean the surface on which the product will be dismantled and use appropriate tools.

### 7.1 Maintenance instructions

 Do not expose the armchair to direct sunlight as this may result in partial discolouration of the fabric.

 Clean mechanical components to limit dust build-up in the frame.

### 7.2 Instructions for maintaining and cleaning the fabric

 Use only fabric cleaning products. Do not saturate the fabric.

 Do not use solvent. Brush if necessary to restore the original appearance.

### 7.3 Daily maintenance

 Before carrying out any maintenance, be sure to follow all safety instructions.

 After any adjustment, repair or servicing work, and prior to use, make sure all screws are correctly tightened.

 If any squeaks or suspect noises can be heard coming from the frame, lubricate or oil linkage points on the frame.

 Cleaning the remote control:  
First disconnect the power supply cable. The remote control can be cleaned with a slightly damp cloth. Do not use a spray. Cleaning agents must not contain acid or salt. Dry the remote control thoroughly before using the armchair again.

## 8. Troubleshooting

The Lift-out chair should not move when in use.

Problem	Solution	Danger
Insufficient space to allow movements of the Lift-out chair	Ensure the Lift-out chair is not positioned too close to the wall, so that the recliner position can be adopted without undue stress on the motor	Blocking of the mechanical system and risk of dysfunction of the motorization
No power	Check that the mains plug is connected	
No power	Check that the remote control is correctly connected and that the cable is not damaged	
No power	Check that the mains cable is not damaged. If it is, replace it	Risk of electrocution
No power	Make sure there are no objects that could impede the armchair's movements	Risk of crushing
No power	Check the power supply to the motor	
No power	Check that electrical components are correctly connected	

## 9. Recycling

This product has been supplied to you by Invacare®, a manufacturer who respects the environment.

It complies with Directive 2002/96/CE on the management of waste electrical or electronic equipment (WEEE).

This product may contain substances which could damage the environment if disposed of inappropriately and not in accordance with legislation in force.

The “crossed-out dustbin” symbol is affixed to this product to encourage you to recycle it at selective collection points.

Be environmentally responsible and recycle this product at the end of its useful life.

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**Invacare® Poirier SAS**
**Invacare® Ireland Ltd**
**Invacare® AB**
**Invacare® B.V.**
**Invacare® Ltd**
Invacare® France Operations SAS