# Invacare® Diana

EN	<b>Bed side rail</b>	
	User Manual .....	1
ES	<b>Barandilla para cama</b>	
	Manual del usuario.....	4
FR	<b>Barrière de lit</b>	
	Manuel d'utilisation .....	6
IT	<b>Sponda laterale del letto</b>	
	Manuale d'uso.....	9
PT	<b>Guarda lateral da cama</b>	
	Manual de utilização .....	12

EN

## I General

### I.1 About this manual

This manual belongs to an Invacare accessory and it contains important information about handling and assembly. In order to ensure safety when using the product, read the user manual of the related product carefully and follow the safety instructions.

### Symbols in this manual

In this user manual, warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

#### WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

#### CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

#### IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

#### Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

### I.2 Intended use

The side rail is intended:

- to prevent the user from falling out of the bed.
- to be operated by a third person and not by the user himself.
- for home care and long term care facilities only.

This manual **MUST** be given to the user of the product. **BEFORE** using this product, read this manual and save for future reference.

#### WARNING!

- Any other or incorrect use could lead to hazardous situations.
- Invacare® accepts no liability for any use, change or assembly of the product other than as stated in this User manual.

### I.3 Service life

The expected service life of this product is five years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

### I.4 Warranty

The warranty covers all material and production defects for two years from the date of delivery, provided it can be demonstrated that such defects were present before delivery. All manufacturing faults or defects must be promptly reported.

Invacare® may repair the fault or replace the component. The warranty provided by Invacare® does not cover additional costs (transport, packaging, labour, sundry expenses, etc. are for the customer's account).

The warranty does not cover:

- Damage caused during transport that is not directly reported to the forwarder at the moment of delivery.
- Repairs performed by unauthorized centers and personnel.
- Parts subject to normal wear.
- Malicious damages or damaged caused by improper use of the bed.

Yes, you can.®

## 2 Safety

### 2.1 General warnings

#### **WARNING!**

##### **Risk of entrapment / suffocation**

- There's a risk of entrapment / suffocation between mattress support, side rail and bed end
- The bed or side rail must not be used for persons under 12 years of age.
  - The bed or side rail must not be used for persons under 45 kg or under 150 cm height.

If the bed is used for restless (spasm) or confused users, there's a risk when the patient operates the bed.

- Make sure that the user can't operate the bed or side rail on his/her own (e.g. disable mechanical and electrical adjustments of the bed functions).

#### **WARNING!**

##### **Risk of user slipping through the openings**

- The bed fulfils all requirements regarding maximum distances. However, it is possible that users with small body dimensions slip through the openings between the side rails or through the opening between the side rail and the mattress support.
- Pay special attention, if the bed is used for the care of users with small body dimensions.

#### **CAUTION!**

##### **Risk of injury**

- There is a risk of entrapment while operating the side rail.
- Pay attention to your fingers.
  - Never force or drop the side rail while handling it.

### 2.2 Mattresses

#### **WARNING!**

##### **Safety aspects regarding combination of side rails and mattresses:**

- In order to obtain the highest possible safety level, when using side rails on the bed, it is important to respect the minimum and maximum measures for mattresses.
- For correct mattress measures; see Bed User Manual for the bed in use.

#### **WARNING!**

##### **Risk of entrapment and/or suffocation**

- The user could get trapped and/or suffocate, if the horizontal space, between the mattress and the inside of the side rail, is too big.
- Follow the minimum width (and length) of mattresses in combination with a given side rail, as stated in the mattress table.
  - Be aware that using very thick or soft mattresses (low density), or a combination of these, increases the risk.

#### **WARNING!**

##### **Risk of falling**


- The user can fall over the edge and get seriously injured, if the vertical distance, between the top of the mattress and the edge of the side rail/bed end, is too short.
- Always keep a minimum distance of 22 cm. The maximum mattress height in combination with a given side rail is stated in the mattress table.

## 3 Setup

### 3.1 Receiving the product

When you receive the product, check the packaging. If the product shows any signs of damage upon delivery, do not use the product.

#### Scope of delivery

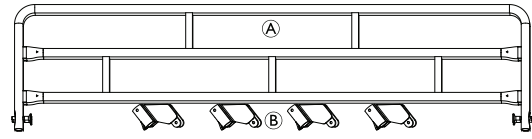- 1 pair of side rails (A)
- 4 brackets (B)
- User manual

### 3.2 Installing the side rail

#### **IMPORTANT!**

- Before installing the side rail, check that your bed configuration is compatible with this side rail. This information is included in your bed user manual.


#### **WARNING!**

##### **Risk of personal injury**


- There is a risk of entrapment or squeezing, while assembling or disassembling the side rail.
- Follow instructions carefully.
  - After each assembly, check that all fittings are properly tightened and that the side rail has the correct function.

1.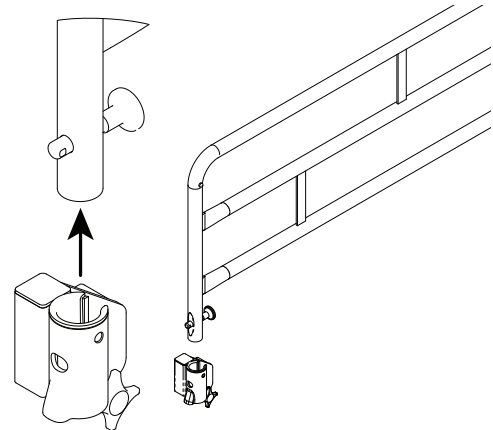


Pull out the pin and mount the bracket on the side rail. Make sure the pin locks the bracket and that the hand wheel is on the outside of the side rail.

2. Do the same thing on the other side.
3. The installation on the bed must be done with the side rail in locked position.
- 4.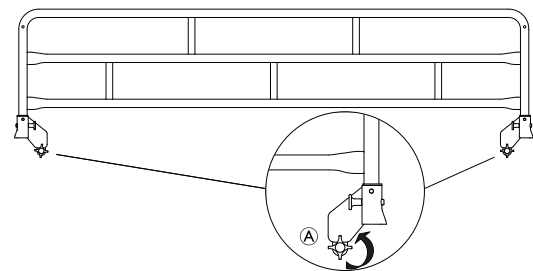


Loosen the knobs (A) on both attachments on the side rail.

5.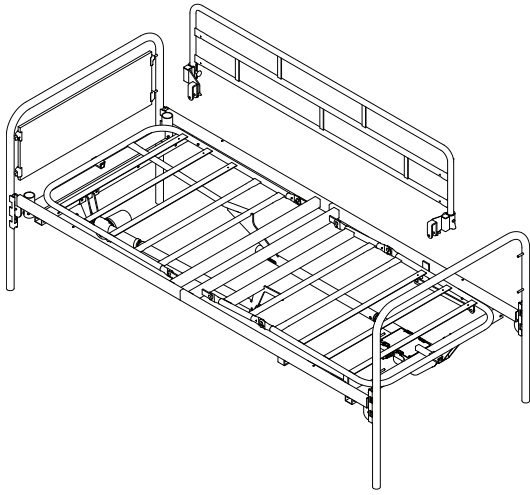

Place the side rail on the frame close to the front bed end.

6.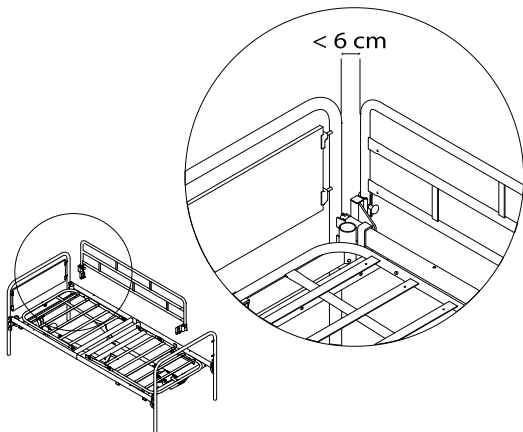

Lower the side rail onto the front leg attachment and tighten with the knob.

7. Repeat the steps for all brackets on both side rails.

**!** **IMPORTANT!**  
Risk of squeezing or getting stuck  
– The distance between the side rail and the bed end must be smaller than 6 cm.

**!** **IMPORTANT!**  
Make sure the side rail is properly locked.  
– Verify that the locking pins are in place and that the knobs are properly tightened.

## 4 Usage

### 4.1 Operating the side rail

**!** **WARNING!**  
**Never leave the user unattended in the bed with the side rail down**  
– Always make sure the side rail is in its highest position and properly locked while leaving the user unattended.


**i** Always stand in the middle of the side rail, when moving it up and down, to ensure that both sides move at the same time.

#### Move the side rail up

1. Lift the side rail up until the locking pin is securely pushed into a housing with an audible click.
2. Make sure the locking pin is securely pushed into a housing.
3. Try moving the side rail up-/downwards to make sure it is properly locked.


#### Move the side rail down

1.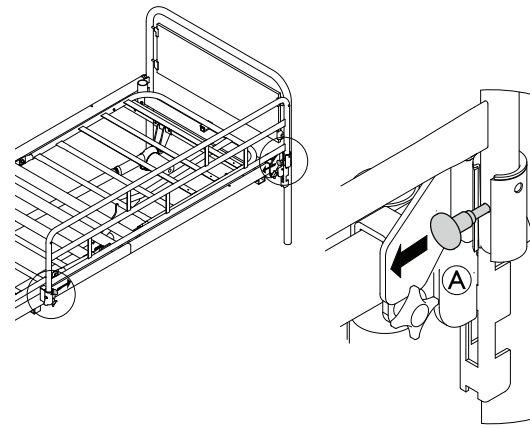

Pull out the locking pins on both sides and push the side rail down, slowly.

**i** Make sure the side rail moves down with the same speed in both sides. If not, the side rail can get stuck. To unlock it; pull it slightly upwards and then push it slowly down again.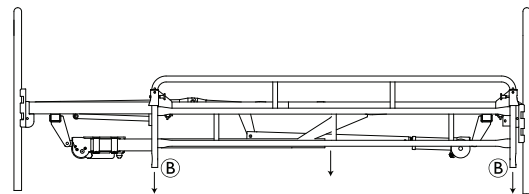

#### **WARNING!**

##### **Risk of squeezing**

There is a risk of getting trapped between the extended tubes on the side rail **B** and the floor or getting squeezed while moving the side rail up and/or down.  
– Be careful while lowering the side rail.

## 5 Maintenance

### 5.1 Cleaning

- Side rails and bed ends should be washed down using a wet sponge, cloth or brush.
- Use ordinary household cleaning agents. Never use acids, alkalines or solvents such as acetone or cellulose thinner.

### 5.2 Waste disposal


Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

Waste disposal must comply with the laws and regulations pertaining waste handling in each country.

## 6 Technical data

### 6.1 Dimensions and weights

All measurements and angles are stated without tolerances, on a non-extended bed.

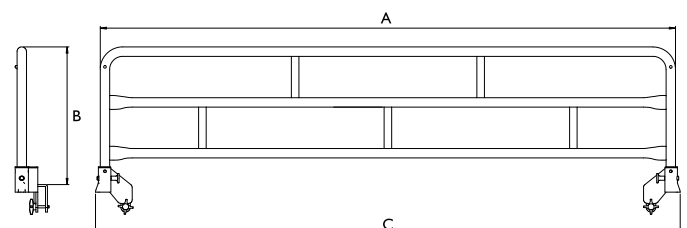A	Length	153 cm
B	Height in upper position	37 cm
C	Length of side rail	155 cm
Total weight (without packaging)		12 kg (pair)
Thickness of mattress		13 - 15 cm

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© Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
Invacare Poirier SAS