Transfer Bench

Assembly, Installation and Operating Instructions

Model No. 9670CE

Yes, you can!

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact Carrier/Dealer for further instruction.

MODEL	DESCRIPTION
9670CE	Assembled with commode

SAFETY SUMMARY

To ensure the safe installation, adjustment and operation of the Transfer Bench, these instructions MUST be followed:

GENERAL WARNINGS

DO NOT install this equipment without first reading and understanding this instruction sheet. If you are unable to understand these **Warnings, Cautions and Instructions**, contact a healthcare professional, dealer or technical personnel if applicable before attempting to install this equipment - otherwise, injury or damage may occur.

SPECIFIC WARNINGS

The weight limitation of transfer bench Manufactured **AFTER** November 1, 1999 is 400 lbs (181.4 kg).

The serial number (date code decal), located on the underside of the seat, is a 5 (five) digit identification and traceability label detailing the date of manufacture, and arranged as follows:

XXXXX 1st digit signifies the last digit of the year.

XXXXX 2nd and 3rd digits signify the month.

XXXXX 4th and 5th digits signify the day.

Example: 91101 = November 1, 1999

Users with limited physical capabilities should be supervised or assisted when using the transfer bench.

Ensure the compression buttons on the back are fully visible through notch on back of seat before using.

Ensure leg extension snap buttons are fully engaged before use.

Check rubber tips for rips, tears, cracks or wear. If any of these conditions exist, replace rubber tips immediately.

SPECIFIC WARNINGS (CONTINUED)

Most tub floors are higher than the bathroom floor. Before using, ensure that the two (2) legs inside the tub are adjusted to the same height and the two (2) legs outside the tub are adjusted to the same height, so that a level seat height adjustment is achieved. DO NOT use if wobbly or unstable.

After ANY adjustments, repair or service and BEFORE use, make sure all attaching hardware is securely tightened - otherwise injury or damage may result.

Tools Required: Phillips Screwdriver, Crescent Wrench, Slotted Screwdriver.

NOTE: The illustrations in this instruction sheet reflect the Transfer Bench without commode. The transfer bench with commode assembles in the same manner.

ATTACHING THE TRANSFER BENCH LEGS (FIGURE 1)

NOTE: Refer to SPECIFIC WARNINGS in the SAFETY SUMMARY of this instruction sheet.

1. Remove all components of the transfer bench and attaching hardware from the packaged container.

NOTE: If your transfer bench is assembled, proceed to ATTACHING THE SEAT ARM in this instruction sheet.

2. Place seat upside down on floor/work bench.
3. Position transfer bench legs on the underside of the seat (snap buttons face inward).

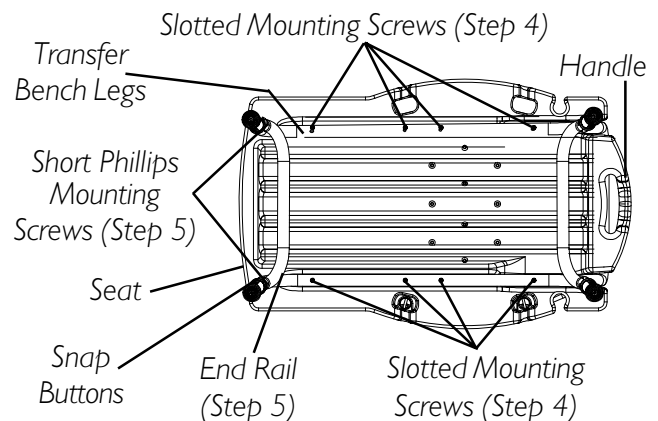
FIGURE 1 - ATTACHING THE TRANSFER BENCH LEGS

4. Secure transfer bench legs to underside of seat with the eight (8) slotted mounting screws provided. Securely tighten.
5. Position the end rail against transfer bench legs, on the opposite side of the handle, and secure with the two (2) short phillips head mounting screws and locknuts provided. Securely tighten.

ATTACHING THE SEAT ARM (FIGURE 2)

NOTE: Refer to SPECIFIC WARNINGS in the SAFETY SUMMARY of this instruction sheet.

1. Place transfer bench in upright position.
2. To attach the seat arm to the transfer bench, perform the following:
 - A. Locate the end rail beneath the soap dish.
 - B. Remove the hardware that secures the end rail to the transfer bench.
3. Position the end rail against the transfer bench leg. Position seat arm on outside of transfer bench leg. Thread the two (2) long phillips head mounting screws through the seat arm, the transfer bench leg, and the end rail. Secure with the two (2) locknuts provided. Securely tighten.
4. Install the back. Refer to INSTALLING/REMOVING THE BACK in this instruction sheet.
5. Adjust the leg extensions, if necessary. Refer to ADJUSTING THE LEG EXTENSIONS in this instruction sheet.

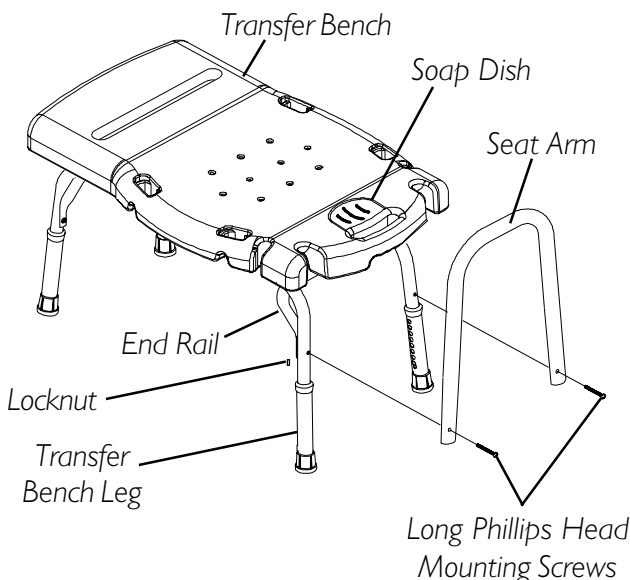
FIGURE 2 - ATTACHING THE SEAT ARM

INSTALLING/REMOVING THE BACK (FIGURE 3)

NOTE: Refer to SPECIFIC WARNINGS in the SAFETY SUMMARY of this instruction sheet.

INSTALLING

1. Position back in slots provided on seat.
2. Push down on back until back is secured.
3. Ensure the compression buttons on the back are fully visible through notch on back of seat before using.

REMOVING

1. To remove back, place one (1) knee on the seat.
2. Hold back handles and lift up.

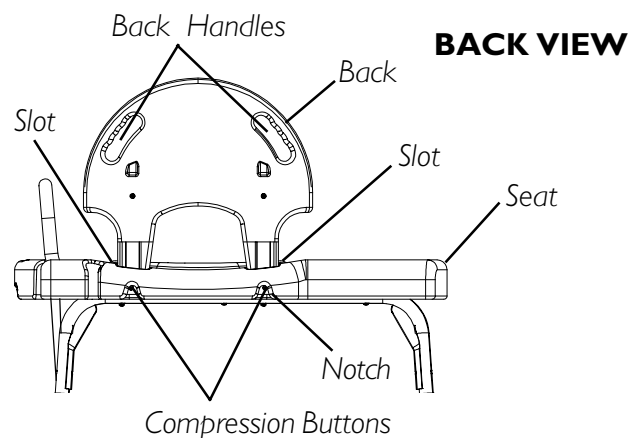
FIGURE 3 - INSTALLING/REMOVING THE BACK

ADJUSTING LEG EXTENSIONS (FIGURE 3)

NOTE: Refer to SPECIFIC WARNINGS in the SAFETY SUMMARY of this instruction sheet.

1. Select one (1) of the nine (9) adjustment holes located on the lower leg frame.
2. Depress the snap button and slide the leg extension up or down to the desired height until snap button protrudes fully through the adjustment hole of the leg extension.
3. Repeat for the other legs.
4. Make sure all four (4) legs are adjusted so that the transfer bench sits level in and outside the tub.


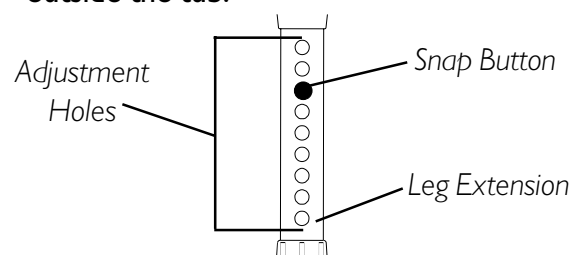
FIGURE 3 - ADJUSTING LEG EXTENSIONS

CLEANING

NOTE: BEFORE using any cleaning products, test an area on the underside of the seat for any discoloration or staining.

- I. Use a mild soap and water or non-abrasive cleanser to clean the Transfer Bench.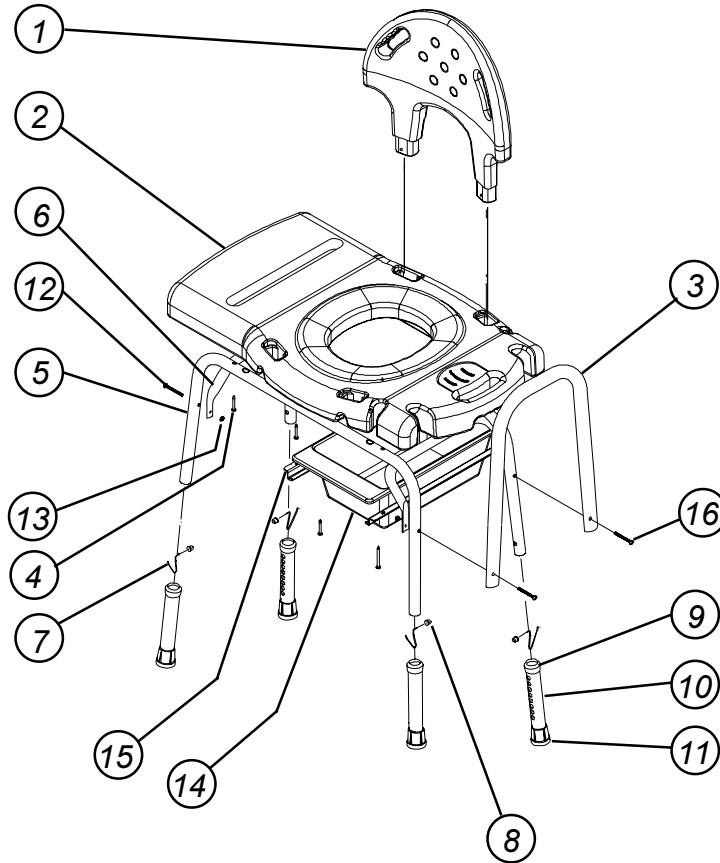

ITEM	DESCRIPTION	QUANTITY
		MODEL NO. 9670CE
1.	SEAT BACK	1
2.	TRANSFER BENCH	1
3.	SEAT ARM	1
4.	10-32 x 1-1/2 PH HD SCREW	8
5.	BENCH LEG	2
6.	END RAIL	2
7.	RELEASE SPRING	4
8.	SNAP BUTTON	4
9.	ANTI-RATTLE	4
10.	LEG EXTENSIONS	4
11.	CRUTCH TIP	4
12.	1/4-20 X 1-1/4 PH SCREW	2
13.	1/4--20 LOCKNUT	4
14.	COMMODOE, PAIL	1
15.	PAIL HOLDER	2
16.	1/4-20 x 1-1/2 PH SCREW	2

LIMITED WARRANTY

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the Transfer Bench to be free from defects in materials and workmanship for one (1) year for the original purchaser/user. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address below, provide dealers name, address, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number. Do not return products to our factory without our prior consent.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, COMMERCIAL OR INSTITUTIONAL USE; PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; OR TO A PRODUCT DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL. SUCH EVALUATION WILL BE SOLELY DETERMINED BY INVACARE. THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE THESE INSTRUCTIONS.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/ PROVINCIAL LAWS AND REQUIREMENTS.

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© 2003 Invacare Corporation
Invacare is a registered trademark of
Yes, you can.™ is a trademark of
Invacare Corporation www.invacare.com
Invacare Poirier S.A.S.
Invacare Corporation.