SMOOV one O10

DE

Neuer Pairing-Prozess ab 06/2020

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EN

New pairing-process since 06/2020

In order to meet the current requirements to protect against cyber-attacks, we have introduced a 128-bit encryption between the SMOOV one O10 and your smartphone.

This means that an individual code is required when your smartphone is for the first time connected to the drive unit. We use a QR code for this.

This code is located on the bottom side of the drive unit and must be scanned during the pairing process.

Proceed as follows:

- Install the SMOOV Mobility App on your smartphone.
- Activate the Bluetooth function on your smartphone.
- Switch the drive unit on using the on/off button.
- Start the SMOOV Mobility App and briefly press the connection icon  in the upper right corner. The connection is established.
- Press "Search SMOOV O10" for the first connection. The address of your SMOOV O10 is displayed. Select it and the camera of your smartphone will be started automatically. Use the camera to scan the QR Code on the bottom of the drive unit.
- Assign and enter a personal PIN of your choice (at least 3 digits) and confirm it.
- Ready. Steady. Go.

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