Quick Reference Guide | Kurzanleitung | Guide rapide |
Brevi istruzioni per l'uso | Manual de empleo conciso |
Beknopte bedieningshandleiding | Breve manual de
instruções | Kortfattad bruksanvisning | Hurtigveiledning

EN

DE

FR

IT

ES

NL

PT

SV

NO

CE 0109
0297

SLS

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Subject to modifications.
All product descriptions are for general information only.
They are not to be understood as a guarantee of quality or durability.

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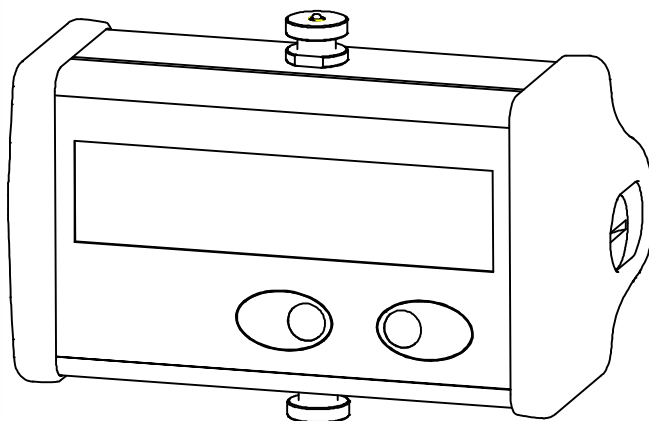
CE 0109
0297

SLS Lifter Scale

1	Safety and general information	3
1.1	Marking on the device	5
1.2	The marking used in this document	5
2	Cleaning, storage and maintenance	6
3	Starting up	7
4	Weighing	8
4.1	Segment test	8
4.2	General weighing information	9
4.3	Weighing when switching the scale on without a patient	9
4.4	Weighing when switching the scale on with a patient	10
4.5	Switching off	11
5	Messages	11
6	Changing the unit (kg ↔ Lb)	12
7	Additional functions (interactive mode)	12
7.1	Setting/changing the switch-off time (Power Off)	12
7.2	Setting/changing the filter	13

1 Safety and general information

This Quick Reference Guide is taken from the detailed *mounting and operating manual*, and is directed at the people who operate the scale. The full mounting and operating manual, I1609, contains relevant information for installation and mounting, as well as for adjustment and subsequent verification.

The SLS lifter scale is a medical device and must only be used by persons whose training or know-how qualifies them to guarantee proper handling.

To avoid putting patients at risk or exposing them to harm through incorrect operation, the operating manual must be read in full before starting up the device, taking note of the information and instructions for proper operation.

Every time the device is used, users must first satisfy themselves that it functions reliably and is in the proper condition.

A visual check of the SLS lifter scale must always be carried out before using the device. It is particularly important to check that

- there is no visible damage, wear or corrosion discernible at the mounting adapters or locating holes,
- the locking cylinders are in place and their fastening bolts are tightened,
- the retaining ring is present in adapters with locating pins.

The measurement results of the scale must be checked for plausibility in connection with the therapy. Check measurements can be carried out with other scales, if necessary.

The user and/or attending physician is solely responsible for deciding whether the actual weighing is an adequate basis on which to make a diagnosis. Conclusions can only be drawn from successive measurements (weight trends) if at least three have been performed, independently of one another. It is not acceptable to start treatment based only on the weight value.

Please note the maximum capacity of the scale (weight of the patient and the patient cradle); this is given on the type plate on the back of the device.

WARNING

A maximum scale loading of 350 kg must *not* be exceeded. It is also necessary to comply with the safety requirements and instructions for appropriate use provided by the lifter and/or equipment manufacturer.

Notice

Once the scale has been mounted on the lifter and aligned, it must not be rotated. This would risk damaging the internal connecting elements! The scale can only be rotated in conjunction with suitable pivot bearings, mounted in the lifter.

Operating conditions

- This device meets the requirements of 2009/23/EC and 93/42/EU, and is radio interference-suppressed in accordance with the applicable harmonized standards EN 60601-1 and EN60601-1-2.
- No devices that emit electromagnetic radiation, such as mobile phones or x-ray equipment, may be operated in the area around the SLS lifter scale. For more detailed information, see mounting and use instructions I1609.
- The SLS lifter scale is not designed for use in a potentially explosive atmosphere. Potentially explosive atmospheres can be produced by the use of flammable anesthetic agents, skin-cleaning agents or skin disinfectants.
- The device must not be modified without the express agreement of the manufacturer. Any modification shall exclude all liability on the part of the manufacturer for any damage resulting therefrom.
- It is forbidden to carry out any repairs on the motherboards, or to replace any components. Should the device malfunction, contact your dealer or the Service Department of HBM.

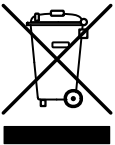
1.1 Marking on the device

CE mark

The CE mark enables the manufacturer to guarantee that the product complies with the requirements of the relevant EC directives (the Declaration of Conformity can be found on the HBM website (www.hbm.com) under HBMdoc).

Statutory waste disposal mark

In accordance with national and local environmental protection and material recovery and recycling regulations, old devices that can no longer be used must be disposed of separately and not with normal household garbage.

If you need more information about disposal, please contact your local authorities or the dealer from whom you purchased the product.

1.2 The marking used in this document

Important instructions for your safety are specifically identified. It is essential to follow these instructions in order to prevent accidents and damage to property.

Symbol	Significance
 WARNING	This marking warns of a <i>potentially</i> dangerous situation in which failure to comply with safety requirements <i>can</i> result in death or serious physical injury.
 CAUTION	This marking warns of a <i>potentially</i> dangerous situation in which failure to comply with safety requirements <i>can</i> result in slight or moderate physical injury.
Notice	This marking draws your attention to a situation in which failure to comply with safety requirements <i>can</i> lead to damage to property.
 Important	This marking draws your attention to <i>important</i> information about the product or about handling the product.

Symbol	Significance
<i>Emphasis</i> See ...	Italics are used to emphasize and highlight text and identify references to sections, diagrams, or external documents and files.
Device -> New	Bold text indicates menu items, as well as dialog and window headings in the program environment.
▶	The triangle at the start of a line indicates instructions to be followed.

2 Cleaning, storage and maintenance

- The device must be protected against moisture penetration, it must not be sprayed. Wipe with a cloth that is only slightly damp, and do not use any abrasive or caustic cleaning agents. Follow the instructions given by the manufacturer of the cleaning agent or disinfectant when using their product.
- If the SLS lifter scale is to be used in areas where there is infection, sterile drapes must be used.
- The SLS lifter scale must be stored somewhere dry, at temperatures between +10°C and +40°C. Condensation must be avoided.

 **CAUTION**

The pressure in batteries increases when they are discharged. So there is an integrated safety valve, which opens when the pressure gets too high. To prevent leakage, batteries must be replaced as soon as the battery symbol appears on the display

- Only ever use the type AA batteries that we have recommended.
- Do not use new batteries together with used ones, and do not mix different types of battery, or batteries of different systems and brands.
- Remove the batteries from the device if it is not going to be used for some time.

- The lifter scale does not need routine maintenance. But it makes sense to check the accuracy, as well as the mechanical connection elements at suitable intervals, in accordance with the inspection plan. If you discover discrepancies in the weight value, or signs of wear on the mounting adapters, immediately take the device out of service and contact your dealer of the Service Department of HBM.

3 Starting up

Unscrew the right-hand cover and insert the battery tray complete with batteries (4 x 1.5 V - type AA), taking care to maintain correct polarity! Push the tray into the battery compartment and screw the cover down again. The scale is now ready to use.

Notice

*Only 1.5 V batteries of type AA must be used.
Do not use rechargeable batteries or a direct voltage source.
Do not touch the contacts.*

Notice

Batteries do not belong in household garbage. Please recycle used batteries at public collection points, or through your retailer.

4 Weighing

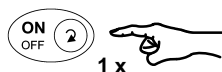
4.1 Segment test

The scale is ready to use about 10 seconds after it has been switched on.

Important

Do not move the scale or load it during this time!


► Press the **ON/OFF** key once to switch the scale on


Display segment test (~5 seconds)

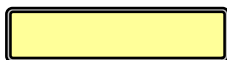
Please watch during the automatic segment test to make sure that as shown, all the segments are on for approx. 5 seconds and off for approx. 2 seconds. If you notice that a segment is faulty, the scale must be repaired.

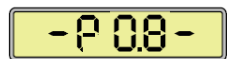Then all the segments go out (~2 seconds)

Display of software used

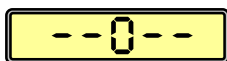Automatic zeroing ¹⁾ (~3 seconds)

¹⁾ If the initial (tare) loading of the scale exceeds 20% of the weighing range, automatic zeroing does not take place. In this case, taring must be done manually (see section 4.4).

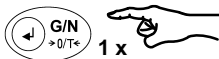
4.2 General weighing information

- The patient to be weighed must sit without support in the load suspension device (e.g. sling). Please note that the permissible total load of the scale must not be exceeded.
- Neither the patient nor the suspension device must be touched during weighing. The cradle should not swing back and forth.
- The weight value should only be read once the scale is at rest. This is indicated when the kg weight unit shows continuously on the display.
- If there is too much patient movement to allow the display to come to a standstill, modifying the filter setting as per *section 7.2* can remedy the situation.


4.3 Weighing when switching the scale on without a patient

Once the segment test is completed as per *section 4.1*, the display is set to 0.0 kg and the scale is ready to use.

Before the patient can sit in the scale, the weight that is not to be included in the measurement (e.g. a towel) must be placed on the scale and tare key G/N ->0/T<- must be pressed.

- ▶ Press the **G/N** key once to tare

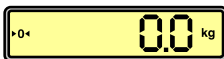You can now weigh

- ▶ Press the **G/N** key once again to toggle between the scale zero setting and the gross weight.
- ▶ Only then should the patient sit in the sling. The weight can now be read on the display.

4.4 Weighing when switching the scale on with a patient

Once the segment test is completed as per *section 4.1*, the scale shows an undefined weight value. (Caution: this is not the patient's weight value)

► Press the **ON/OFF** key once to switch the scale on

Display segment test (~5 seconds)

Then all the segments go out

Followed by the display of software used

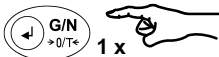
The total weight is displayed (with the cradle!)

Notice

If zero is displayed at this point, it is not possible to weigh like this, see section 4.3 for how to continue.

- Now press the tare key to determine the weight of the patient. Then the patient gets out of the scale (the display value is now negative)
The weights of the sling, towel, etc., that are not to be measured, stay on the scale or must be re-attached.


The value that is now displayed without the minus sign corresponds to the actual body weight of the patient.

► Press the **G/N** key once to tare

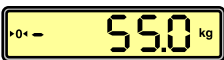The total weight is set to zero

Once the patient gets out of the cradle, their weight is displayed with a negative sign.

4.5 Switching off

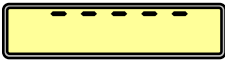
1 x

- ▶ Pressing the **ON/OFF** key once switches the scale off.

5 Messages

Overload

Scale overloaded! Please note the maximum capacity of the scale (see type plate on the back of the device).

Underload

2 x

- ▶ Press the **ON/OFF** key twice to switch the scale off and then on again.

Battery status

The batteries are flat and must be replaced!

6 Changing the unit (kg ↔ Lb)

Important

Only possible for scales that are not legal for trade/not verified.

- ▶ Switch the scale on and wait for ~ 10 seconds (see section 4.1).

Unit kg

- ▶ Keep it pressed for ~ 5 seconds

Unit lb

Proceed in the same way to change from lb to kg. Whichever unit is last set is retained even when the scale is switched off.

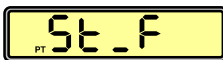
7 Additional functions (interactive mode)

7.1 Setting/changing the switch-off time (Power Off)

- ▶ Switch the scale on and wait for ~ 10 seconds (see section 4.1).

- ▶ Simultaneously press the **ON/OFF** and **G/N** keys to activate interactive mode.

Interactive mode (Standard_Function)

Press the **G/N** key once again to activate the automatic switch-off time (Power Off) setting

- ▶ Make the selection by *repeatedly* pressing the **ON/OFF** key (the number flashes):

Manual power off only (with the **ON/OFF** key)

Power off *automatically after 30 seconds* (only when the scale is not loaded)

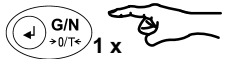
Power off *automatically after 60 seconds* (only when the scale is not loaded)

Power off *automatically after 5 minutes* (only when the scale is not loaded)


Power off *automatically after 10 minutes* (only when the scale is not loaded)

► Confirm your selection with the **G/N** key

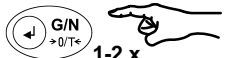At this point, you can check *PO* or change it again by pressing **ON/OFF** once again.

► Press the **G/N** key once to go to filter setting **AF**

or

► Press the **G/N** key twice to go back to weighing mode

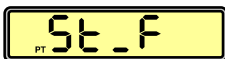
7.2 Setting/changing the filter

The SLS lifter scale has the option to filter the weight display by making relevant settings.


► Switch the scale on and wait for ~ 10 seconds (see section 4.1).

► *Simultaneously* press the **ON/OFF** and **G/N** keys to activate interactive mode.


Interactive mode (Standard_Function)

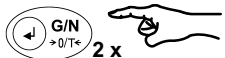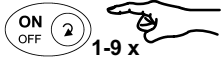▶ Press once again to activate setting the filter setting (Amplifier Filter).

▶ Make the selection by *repeatedly* pressing the **ON/OFF** key (the number flashes):

0 = Filtering switched off

1 = Weak filtering:
 + very fast weight display
 - unsteady display when there are movements

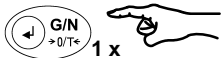
4 = Medium filtering (recommended):
 + fast weight display
 + steady display

⋮
 ⋮
 ⋮


8 = Very strong filtering:
 + possible to weigh fidgety persons
 - long waiting time before the scale weight or zero point display is steady

▶ Confirm your selection and adopt the settings by pressing the **G/N** key.

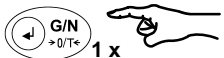At this point, you can check *AF* or change it again by pressing **ON/OFF** once again.

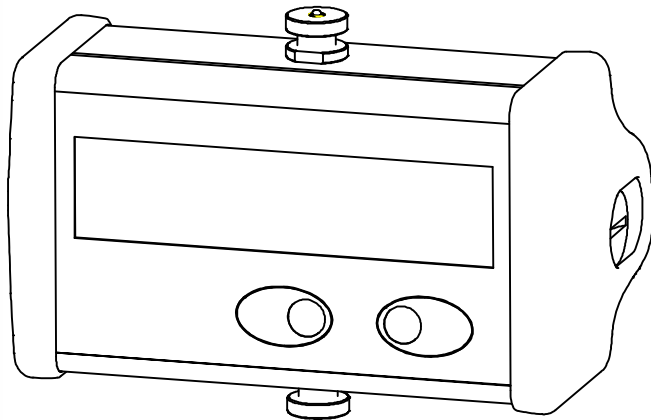
▶ Press the **G/N** key once to go back to weighing mode.

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© Hottinger Baldwin Messtechnik GmbH.