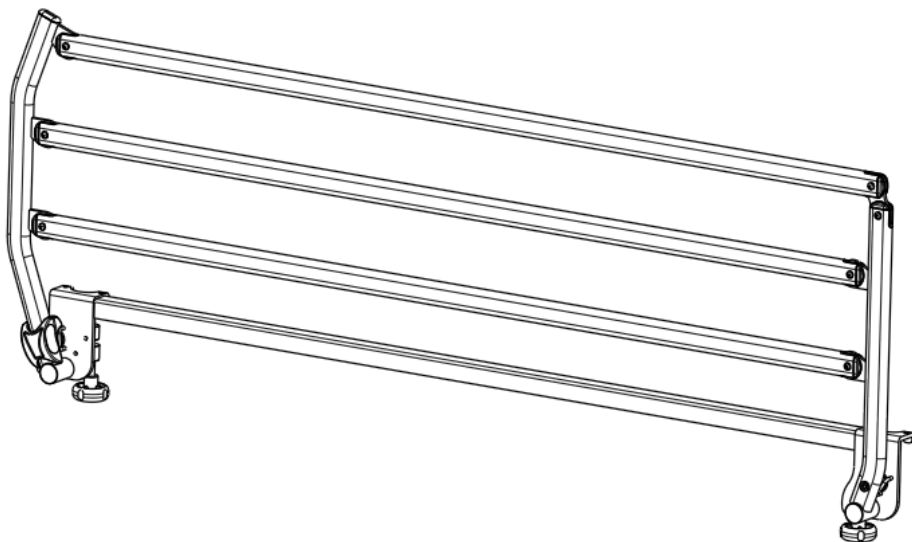
Yes, you can.®

Invacare® **Verso™ II**

User manual	EN
Manuel d'utilisation	FR
Gebruiksaanwijzing	NL
Manual de Utilização	PT
Manual del usuario	ES
Manuale d'uso	IT
Bruksanvisning	SV

SIDERAILS FOR *Invacare*® BEDS ONLY

ALEGIO™
SB®
SONATA™

Thank you for choosing *Invacare*®'s **Verso™II** folding siderail.

The *Invacare*® **Verso™II** is especially designed to prevent the patient to fall out of the bed.

The **Verso™II** siderail offers easy handling and increased security for the patient. The **Verso™II** is designed to operate the siderails by a third person. (Personal Looking after, Aide in Residence, family...) and not by the patient himself. The intended use of these side rails is for home care only!

Invacare® is certified according to EN ISO 9001 and ISO 13485 which ensures that our customers are always supplied with products of uniform quality.

Throughout the entire production process, our materials and components are quality controlled by the operators. A final test is made when the product is fully assembled. If the product does not correspond to the quality demands of *Invacare*®, it will not pass quality control.

In the event of a problem in connection with the delivered product, please contact your local *Invacare*® supplier.

Invacare® will take no responsibility if the product is used or assembled in any way other than stated in this user manual. Only accessories recommended by *Invacare*® may be used on the **Verso™II** siderail. This product must only be operated by qualified staff who has received the necessary instruction and training.

Invacare® recommends that the average usable life of this product is ten years, providing the product has been correctly maintained according to the manufacturer's recommendations.

Please read this user manual carefully before using the siderail.

1. General

- **Verso™II** is CE-marked in accordance with directive 93/42/EEC concerning medical devices.
- **Verso™II** has been tested and approved according to EN 1970-A1 & EN 60601-2-52.
- **Verso™II** has undergone a risk analysis according to EN/ISO 14971.

Symbols

The product should be recycled where possible:

Refer to user's manual:

2. Safety

Please be aware of the following possible risks when using the siderail:

The use by patients with an atypical anatomy can create risks of injury.

This siderail was designed for patients with a typical grown-up size, not to use this product for children under age 12 or for patients with an atypical anatomy. Every child under age 12 near this product must be watched to avoid any potential risk of accident.

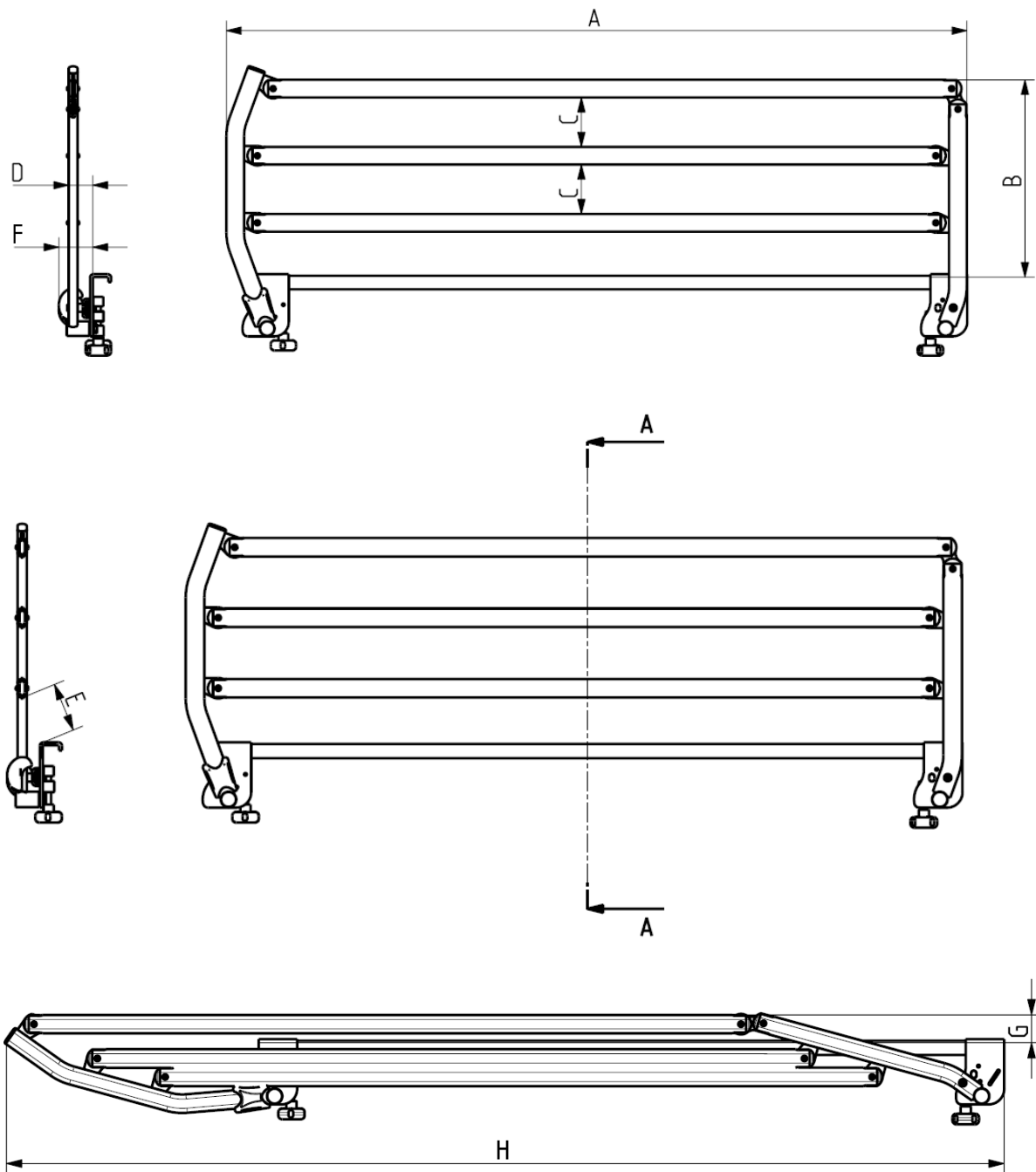
The side rail must be carefully manipulated if the patient is in the bed.

Invacare® accepts no liability for any use, change or assembly of the **Verso™II** siderail other than mount on the bed, as stated in this user manual.

There is a risk of squeezing and entrapment during operation of folding and unfolding.

Never force or drop the side rail during handling.

3. Technical data

General specifications

A → Length: 1500mm

B → Height: 400mm

C → Space between bars: 100mm

D → External dimension out bed: 50mm

E → Space between bed and bar: 100mm

F → External overall dimension out bed: 70mm

G → Height in folding position: 55mm

H → Length in folding position: 1940mm

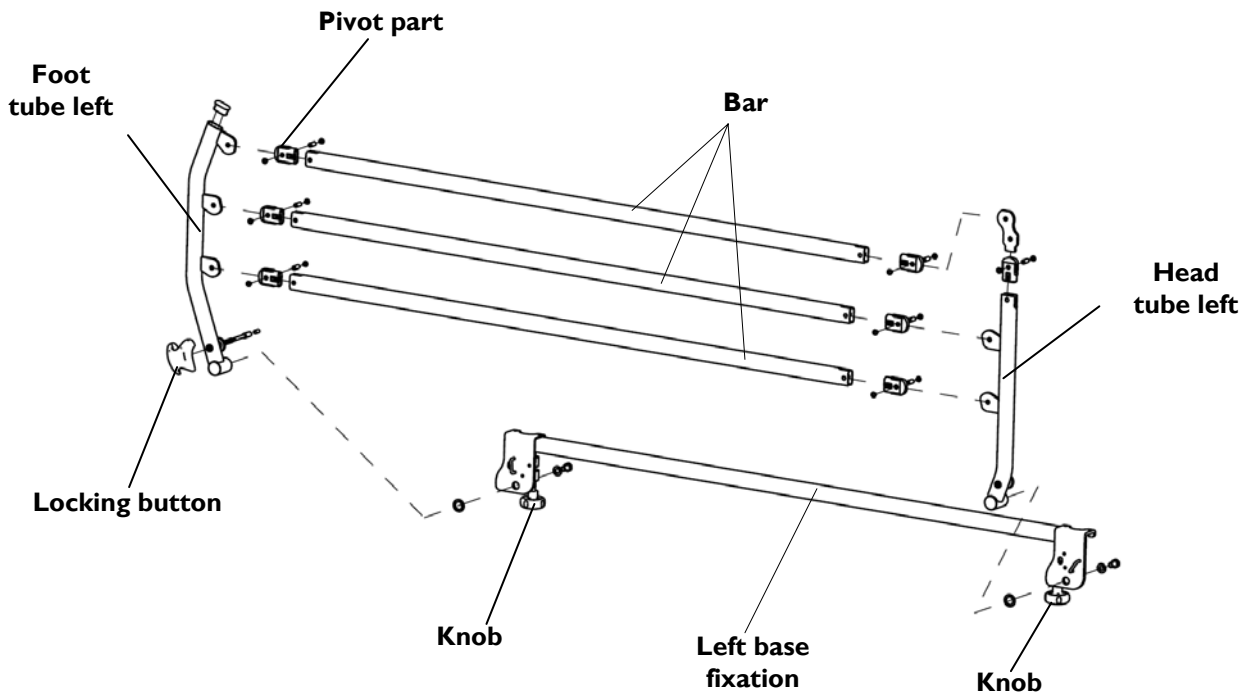
Operating forces of buttons Max: → 40N

Total weight (weight out of cardboard box) → 9.3Kg

Thickness of the mattress mini: 100mm

maxi: 180mm

4. Components and their functions

Accessories

It is compulsory to use original *Invacare*® Accessories which you can purchase through any *Invacare*® dealer. For repair, please contact your local *Invacare*® dealer. Service and repair must be carried out by a properly trained technician.

An adaptation kit (1557395) is available on demand for Alegio beds built before November 2009.

5. Set up and installation on Invacare® Beds only

Unpacking

Should the packaging be damaged upon receipt, each side rail must be examined for visible defects. In the event of any damage, please contact *Invacare*®.

Assembly instructions

The siderails are provided completely ready to be used and installed on the bed. No tool is necessary to its installation.

1. The installation on the bed must be done with the siderail in locked position.
2. Place both jaws of the siderail on the frame of the bed, by positioning the locking button in footboard. The jaw opposite straddling the lifting pole support to assure a lower space of 60mm (< 60mm) between the siderail and the bed end head (see illustration A below).
3. For beds equipped with bed ends with fast locking, it is beforehand necessary to pull and turn $\frac{1}{4}$ of tour the trigger, previously positioned the jaw at the head of bed under the trigger and relocks it after. (see illustration B below).
4. Once jaws positioned well on the frame of bed, fix the siderail by using both knobs and tightened them firmly.

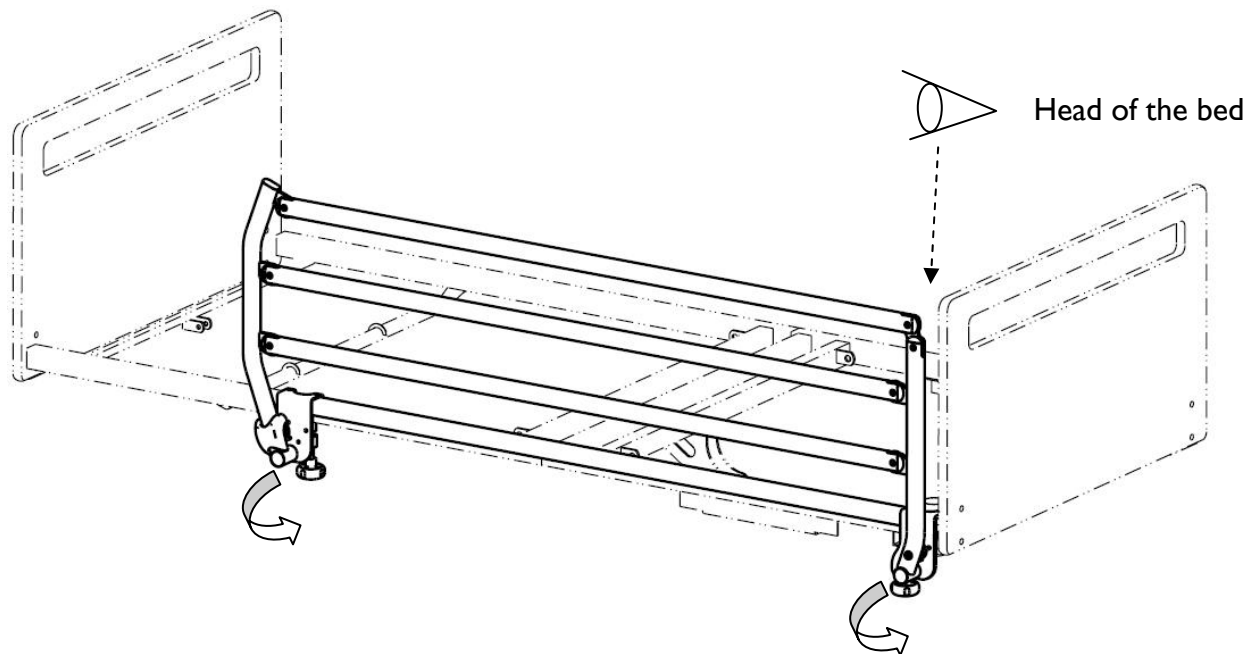
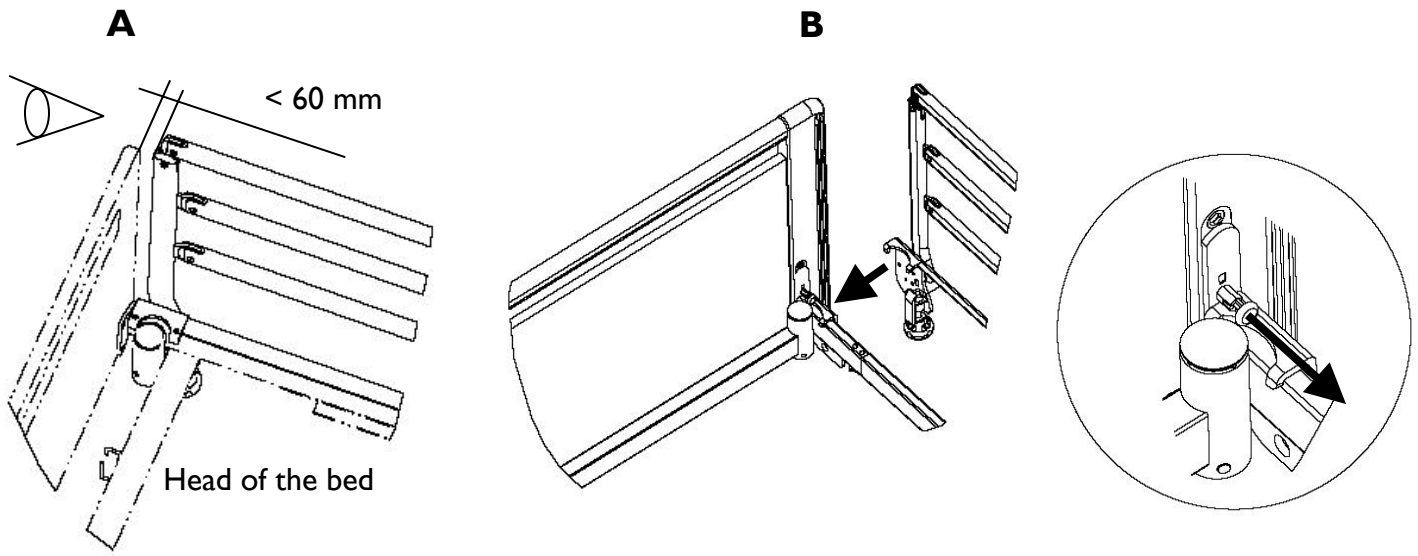
Warning: not tighten and/or an insufficient tightening can entail a risk of fall of the siderail and/or of the patient.

5. Make some manipulations to verify the smooth running (folding / unfolding, locking / unbolting, tightening ...).

Warning: make sure of its perfect locking by applying a longitudinal alternate movement in the siderail itself and verify that the locking pin is positioned well in its housing.

6. Instructions of use

To open (fold the siderail):

Pull the locking button and start the folding movement with one hand on the upper horizontal bar. Release the locking button during the folding movement and handling the siderail until its lower position (under the mattress).

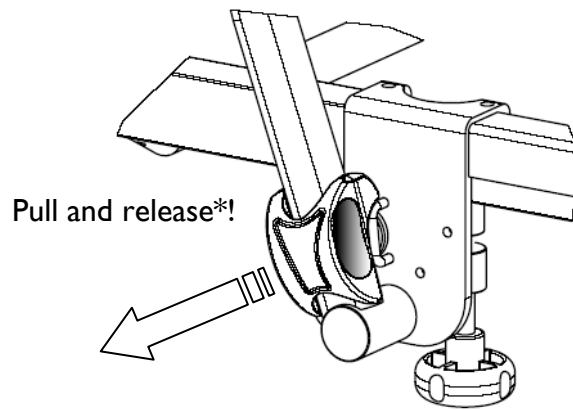
Take care to don't squeeze body of patient during this step.
On Medley Low version, there is a risk of feet pinching with the locking jaws due to the lowered position of the bed frame. Adjust the position of feet accordingly.
To avoid all finger pinch out during the manipulation, fingers must be positioned in the two lateral dug areas designed for that purpose, in locking button (grey zone, next page).

To close (unfold the siderail):

Pull the siderail by the upper bar and rising until it engage in the locking system.

Take care to don't squeeze body of patient during this step.

* Release just after started to fold the siderail

Warning: make sure of its perfect locking by applying a longitudinal alternate movement. In the siderail itself and verify that the locking pin is positioned well in its housing. Always verify the good locking of the siderail before leaving unsupervised the unattended patient.

7. Maintenance and cleaning

Maintenance

- The siderails **Verso™II** are Epoxy steel and do not require particular maintenance. After 3 months of use, control their fixing, the system of locking and its mechanism. In the event of normal use, the first control of maintenance must be after 2 years then each year.
- When you lower the siderail, accompany its opening to ensure a longer life of product.

Cleaning

- To clean, use a sponge or a wet rag. Use ordinary detergents and dry it.
- Never use acids, alkaline or solvents such as the cellulose acetone.

Attention

- Check that the siderails were correctly locked.
- During the assembly and use of the siderails, take careful to do not grip the fingers.

Assistance

- In the case where the provided product would present anomalies, Please contact your supplier **Invacare®**
- **Invacare®** cannot be held responsible for damage caused by a incorrect or not indicated in this user manual.

Warranty

The siderails **Verso™II** are guaranteed 2 years after its selling. The warranty is applied only for manufacturing defects.

Restrictions

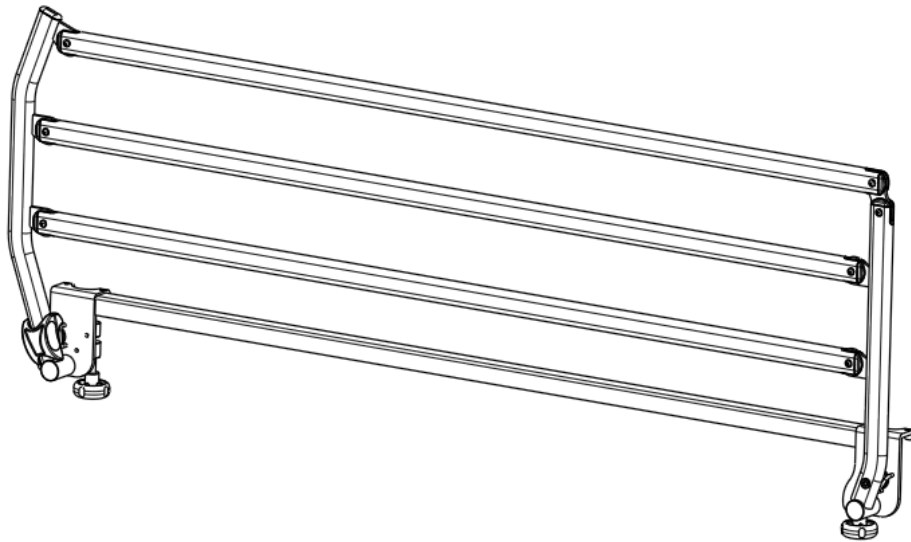
The warranty does not cover the following situations:

- Storage or misuse.
- Fair wear by the normal use of the product.
- Bad or wrong maintenance.
- Repair or modification of the product by other none authorized people.

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Invacare® GmbH
Invacare® Austria GmbH
Invacare® Poirier SAS
Invacare® Ireland Ltd
Invacare® AB
Invacare® B.V.