Invacare® Accessories

en	Lifting Pole User Manual	2
de	Aufrichter Gebrauchsanweisung	3
es	Incorporador Manual del usuario	4
fi	Nousutuki Käyttöohje	6
fr	Potence Manuel d'utilisation	7
it	Alzamalati Manuale d'uso	8
nl	Papegaaï Gebruiksaanwijzing	10
no	Løftebøyle Bruksanvisning	11
pt	Pendural Manual de utilização	13

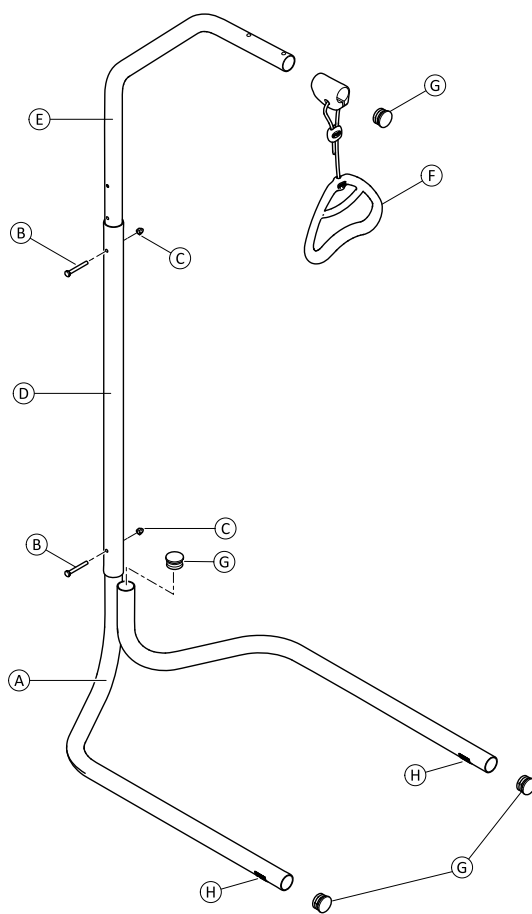
Fig. 1

This manual MUST be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

Introduction

Thank you for choosing an Invacare product.

This user manual contains important information about the handling of the product. In order to ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Warranty information

Terms and conditions of the warranty are part of the general terms and conditions particular to the individual countries in which this product is sold.

Contact information for your local Invacare office is located at the end of this manual.

Limitation of liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorized modifications and/or use of unsuitable spare parts

Safety

WARNING! Risk of injury

- Always make sure that the lifting pole is positioned correctly and standing on a flat surface.
- Do not use the lifting pole with beds that have remote controlled height adjustment.
- Do not make any adjustments while the lifting pole is in use.
- After each adjustment of the lifting handle, check that the cord and the lifting handle are secured.
- Take particular care when using a manual or electric head section to assist in lifting the patient.
- Remove the lifting pole from the bed area before using a lifter to move the patient from or to the bed.
- Do not alter any part of the lifting pole. Only use original spare parts from Invacare.
- After each assembly check that all fittings are properly tightened and all parts have the correct function.

Intended use

The lifting pole is intended to help users raise the upper part of their bodies while lying in a nursing bed and to be used only with beds that don't have remote controlled height adjustment.

Labels and symbols on the product

Product Label

Symbols

	Serial Number
	Reference Number
	Manufacturer Address
	Manufacturing Date
	Max. Safe Working Load
	Read user manual
	This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

Assembly (Fig. 1)

Two 10 mm spanners

- Ⓐ Base (1 pc)
- Ⓑ Screw TE M6x50 (2 pcs)
- Ⓒ Cap nut M6 (2 pcs)
- Ⓓ Connecting tube (1 pc)
- Ⓔ Shaped tube (1 pc)
- Ⓕ Lifting handle with cord (1 pc)
- Ⓖ Cap Ø40 (4 pcs)
- Ⓗ Base cap (2 pcs)

1. Fix the base Ⓐ to the connecting tube Ⓓ using one screw Ⓑ and cap nut Ⓒ.
2. Fix the shaped tube Ⓔ to the connecting tube Ⓓ, using the second screw Ⓑ and cap nut Ⓒ. Select the most suitable hole according to the required height.
3. Position the lifting handle Ⓕ, so that the rubber reinforcement remains blocked between the two pins on the top shaped tube.

Usage

1. Place the lifting pole at the head end of the bed with the base fully under the bed.
2. Adjust the length of the cord so that the patient can grip the lifting handle with both hands.
 - a. Loosen the cord and hold out while sliding the plastic cord lock upwards or downwards until the handle reaches the preferred height.
 - b. Lock the handle by pressing back the cord in the lock and pull the handle downwards.
 - c. Check that the two cords above the cord lock are parallel and make sure the cord is properly locked by pulling the handle hard.
3. You can raise the head section of the bed to make it easier for the user to reach the lifting handle.

Maintenance and cleaning

Before each use

- Check that screws and nuts are firmly tightened and the cord doesn't show any signs of damages.

Every second year

The product does not require any special maintenance, although we recommend to make regular checks at least every two years:

- Check that screws and nuts are firmly tightened.
- Check that the tubes are in the correct position.
- Check that the cord doesn't show any signs of damages.
- Check the connections and attachments.
- Visually inspect the lifting pole for any damages.

If you detect any damages or issues, contact your distributor or Invacare representative.

Cleaning

- Clean the product with a moist cloth or sponge and wipe dry immediately.
- Use normal household detergents for cleaning. Do not use acids, alkalines or solvents such as acetone or cellulose thinner.
- Do not clean the product in automatic washing plants or with water-jet based cleaning equipment.

Technical data

Adjustable height	182 cm to 200 cm
Lifting handle adjustable height	122 cm to 164 cm
Safe working load	80 kg
Weight	7 kg
Frame material	Chromium-plated steel tube
Base material	Painted steel tube

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© 2017 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
Invacare AB